



CORFLEX

**CORFLEX IS A GLOBALLY BASED
MANUFACTURER AND DISTRIBUTOR OF QUALITY
ORTHOPEDIC AND REHABILITATION PRODUCTS**



KNEE MISC

CRYOTHERM VARI WRAP

- > Accommodates various parts of the body and provides compression and cold/hot therapy
- > Indicated for post-op use, arthritis, strains/sprains, and for integration into rehab program
- > Manufactured from a soft breathable foam laminate with contact closure and pocket to accommodate reusable gel packs



PRODUCT NUMBER

0 GELS	1 GEL	2 GELS
72-7070	72-7071	72-7072

SIZING GUIDE

UNIV

Fits up to 40" circumference

Additional gels sold separately:
72-7001 - Single 5 1/2" x 11" Gel
72-7000 - Case/10 5 1/2" x 11" Gel

APPLICATION INSTRUCTIONS

1. Observe warnings and instructions for use of gel packs.
2. Place one pack at a time into pocket and secure in pocket.
3. Body of wrap is in two pieces, you may use both or one as desired.
4. Place wrap on affected area and secure to desired comfort level.

CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



TELEPHONE

1-800-426-7353, Ext1



FAX

1-603-623-4111



E-MAIL

Service@corflex.com



MAIL

CORFLEX, Inc.
Attn: Customer Service
669 East Industrial Park Dr
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.