



CORFLEX
.....

**CORFLEX IS A GLOBALLY BASED
MANUFACTURER AND DISTRIBUTOR OF QUALITY
ORTHOPEDIC AND REHABILITATION PRODUCTS**



CERVICAL SUPPORTS

CRYOTHERM NECK WRAP

- > Anatomical design provides concentrated cold/hot therapy for the cervical spine
- > Indicated for post-op use, arthritis, strains or sprains and for integration into rehab program
- > Manufactured from a soft breathable foam laminate with contact closure and pocket to hold reusable gel pack



PRODUCT NUMBER

	0 GELS	1 GEL	2 GELS
3 1/2"	72-7120	72-7121	72-7122
4 1/2"	72-7130	72-7131	72-7132

SIZING GUIDE

Fits up to a 22" neck circumference

Additional gels sold separately:
72-7003 - Single 3 1/2" x 11" Gel
72-7002 - Case/10 3 1/2" x 11" Gel

APPLICATION INSTRUCTIONS

1. Observe warnings and instructions for use of gel packs.
2. Place one gel pack at a time into pocket and close with Velcro closure.
3. Locate wrap at base of neck, placing gel over affected area.
4. Wrap strap around neck loosely and attach strap.

CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Remove any metal stays if possible, close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



TELEPHONE

1-800-426-7353, Ext1



FAX

1-603-623-4111



E-MAIL

Service@corflex.com



MAIL

CORFLEX, Inc.
Attn: Customer Service
669 East Industrial Park Dr
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.