



**CORFLEX**


**CORFLEX IS A GLOBALLY BASED  
MANUFACTURER AND DISTRIBUTOR OF QUALITY  
ORTHOPEDIC AND REHABILITATION PRODUCTS**



**FOOT / ANKLE MISC**

**CRYOTHERM ANKLE WRAP**

- > Anatomical design provides compression and concentrated cold/hot therapy for the ankle
- > Indicated for post-op use, arthritis, strains/sprains and as part of rehab program
- > Manufactured from a soft breathable foam laminate with contact closure and pockets to hold reusable gel packs

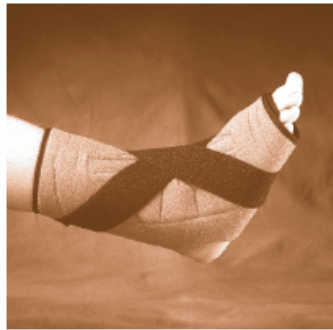

**PRODUCT NUMBER**

0 GELS	2 GELS	4 GELS
72-7140	72-7142	72-7144

**SIZING GUIDE**

**UNIV**

Fits up to 14" ankle circumference

Additional gels sold separately:  
72-7003 - Single 3 1/2" x 11" Gel  
72-7002 - Case/10 3 1/2" x 11" Gel

Measurement is circumference from base of heel around ankle

**APPLICATION INSTRUCTIONS**

1. Observe warnings and instructions for use of gel packs.
2. Place one gel pack at a time into each pocket and secure in pocket.
3. Place heel in opening, ensuring label is at top of wrap.
4. Working from bottom to top, secure straps.
5. Grasp both 2" stretchable straps, cross over in front, pull to rear and attach for a comfortable fit.

**CARE & CLEANING**

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

**HOW TO PLACE AN ORDER**

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



**TELEPHONE**  
1-800-426-7353, Ext1



**FAX**  
1-603-623-4111



**E-MAIL**  
Service@corflex.com



**MAIL**  
CORFLEX, Inc.  
Attn: Customer Service  
669 East Industrial Park Dr  
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109  
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.