



**CORFLEX**

**CORFLEX IS A GLOBALLY BASED  
MANUFACTURER AND DISTRIBUTOR OF QUALITY  
ORTHOPEDIC AND REHABILITATION PRODUCTS**



## WRIST SPLINTS

### CRYOTHERM WRIST WRAP

- > Anatomical design provides compression and concentrated cold/hot therapy for the wrist
- > Indicated for post-op use, arthritis, strains/sprains and for integration into rehab program
- > Manufactured from a soft breathable foam laminate with contact closure and pockets to hold reusable gel packs



#### PRODUCT NUMBER

0 GELS	1 GEL	2 GELS
72-7150	72-7151	72-7152

#### SIZING GUIDE

**UNIV**

Fits up to 9" wrist circumference

Additional gels sold separately:  
72-7003 - Single 3 1/2" x 11" Gel  
72-7002 - Case/10 3 1/2" x 11" Gel

### APPLICATION INSTRUCTIONS

1. Observe warnings and instructions for use of gel packs.
2. Place one gel pack at a time into pocket and secure in pocket.
3. Slide hand into product, place thumb through thumb hole.
4. Wrap the two stretchable straps around the product, thread through the buckles and secured to desired comfort level.

### CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Remove any metal stays if possible, close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

### HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



**TELEPHONE**

1-800-426-7353, Ext1



**FAX**

1-603-623-4111



**E-MAIL**

Service@corflex.com



**MAIL**

CORFLEX, Inc.  
Attn: Customer Service  
669 East Industrial Park Dr  
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109  
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.