



CORFLEX

**CORFLEX IS A GLOBALLY BASED
MANUFACTURER AND DISTRIBUTOR OF QUALITY
ORTHOPEDIC AND REHABILITATION PRODUCTS**



WRIST SPLINTS

ULTRA FIT WRIST SPLINT

- > Features easy one-hand application that provides comfortable support of the wrist while preserving finger mobility
- > Indicated for Carpal Tunnel Syndrome, tendonitis, wrist strains or sprains, and post-cast removal
- > Manufactured from durable, lightweight material with removable malleable aluminum palmer stay, flexible dorsal stay and contact closure



PRODUCT NUMBER

	UNIV	XS	S	M	L	XL
8" Right	73-1000	73-1002	73-1003	73-1004	73-1005	73-1006
8" Left	73-1001	73-1012	73-1013	73-1014	73-1015	73-1016

	UNIV	XS	S	M	L	XL
10" Right	73-1040	73-1042	73-1043	73-1044	73-1045	73-1046
10" Left	73-1041	73-1052	73-1053	73-1054	73-1055	73-1056

SIZING GUIDE

	UNIV	XS	S	M	L	XL
	5"-9"	4"-5"	5"-6"	6"-7"	7"-8"	8"-9"

Measurement taken from wrist circumference

APPLICATION INSTRUCTIONS

1. Open wrist splint and lay flat on clean surface with inside facing up.
2. Place hand on splint so that the wide stay is comfortably placed in your palm (stay may be contoured to match the curvature of your wrist).
3. Move stay assembly and place on top of splint so stay is centered on top of wrist and comfortably closes around wrist.
4. Bring strap with hook closure between index finger and thumb and attach to loop closure on top of splint.
5. Starting a wrist, thread side straps through plastic buckles and pull snugly and attach hook and loop closure to desired comfort level.

CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Remove any metal stays if possible, close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



TELEPHONE

1-800-426-7353, Ext1



FAX

1-603-623-4111



E-MAIL

Service@corflex.com



MAIL

CORFLEX, Inc.
Attn: Customer Service
669 East Industrial Park Dr
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.