



**CORFLEX.**

**CORFLEX IS A GLOBALLY BASED  
MANUFACTURER AND DISTRIBUTOR OF QUALITY  
ORTHOPEDIC AND REHABILITATION PRODUCTS**



**WRIST SPLINTS**

**ULTRA FIT COOL WRIST SPLINT**

- > Features easy one-hand application that provides cool and comfortable support of the wrist using breathable spacer material
- > Indicated for Carpal Tunnel Syndrome, tendonitis, wrist strains or sprains, and post-cast removal
- > Manufactured from durable, lightweight spacer fabric with removable malleable aluminum palmer stay, flexible dorsal stay and contact closure



**PRODUCT NUMBER**

	<b>XS</b>	<b>S</b>	<b>M</b>	<b>L</b>	<b>XL</b>
8" Right	73-1022	73-1023	73-1024	73-1025	73-1026
8" Left	73-1032	73-1033	73-1034	73-1035	73-1036

**SIZING GUIDE**

<b>XS</b>	<b>S</b>	<b>M</b>	<b>L</b>	<b>XL</b>
4"-5"	5"-6"	6"-7"	7"-8"	8"-9"

Measurement taken from wrist circumference

**APPLICATION INSTRUCTIONS**

1. Release all hook and loop closures.
2. Slide hand into splint so wide stay fits comfortably in palm (Note: stay may be shaped to match curvature of wrist).
3. Close all hook and loop closures to desired tension for comfortable fit.

**CARE & CLEANING**

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Remove any metal stays if possible, close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

**HOW TO PLACE AN ORDER**

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



**TELEPHONE**

1-800-426-7353, Ext1



**FAX**

1-603-623-4111



**E-MAIL**

Service@corflex.com



**MAIL**

CORFLEX, Inc.  
Attn: Customer Service  
669 East Industrial Park Dr  
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109  
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.