



**CORFLEX**

**CORFLEX IS A GLOBALLY BASED  
MANUFACTURER AND DISTRIBUTOR OF QUALITY  
ORTHOPEDIC AND REHABILITATION PRODUCTS**



## THUMB SPLINTS

### ULTRA FIT COOL WRIST SPLINT W/ABDUCTED THUMB

- > Features easy one-hand application and provides cool and comfortable thumb and wrist immobilization using breathable spacer fabric
- > Indicated for de Quervain's Syndrome, tendonitis, thumb strains or sprains and post-cast removal
- > Manufactured from durable, lightweight spacer fabric with removable malleable aluminum palmer and thumb stays, flexible dorsal stay and contact closure



#### PRODUCT NUMBER

	XS	S	M	L	XL
8" Right	73-5322	73-5323	73-5324	73-5325	73-5326
8" Left	73-5332	73-5333	73-5334	73-5335	73-5336

#### SIZING GUIDE

XS	S	M	L	XL
4"-5"	5"-6"	6"-7"	7"-8"	8"-9"

Measurement taken from wrist circumference

#### APPLICATION INSTRUCTIONS

1. Release all hook and loop closures.
2. Slide hand into splint so wide stay fits comfortably in palm (Note: stay may be shaped to match curvature of wrist).
3. Close all hook and loop closures to desired tension for comfortable fit.

#### CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Remove any metal stays if possible, close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

#### HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



**TELEPHONE**

1-800-426-7353, Ext1



**FAX**

1-603-623-4111



**E-MAIL**

Service@corflex.com



**MAIL**

CORFLEX, Inc.  
Attn: Customer Service  
669 East Industrial Park Dr  
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109  
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.