



CORFLEX

**CORFLEX IS A GLOBALLY BASED
MANUFACTURER AND DISTRIBUTOR OF QUALITY
ORTHOPEDIC AND REHABILITATION PRODUCTS**



WRIST SPLINTS

ELASTIC WRIST SPLINT

- > Contoured design allows for optimal finger mobility while providing moderate wrist support
- > Indicated for Carpal Tunnel Syndrome, tendonitis and wrist strains or sprains
- > Manufactured from premium grade woven-cotton elastic with palmer stay and contact closure



PRODUCT NUMBER

	XS	S	M	L	XL
8" Right	73-5770	73-5771	73-5772	73-5773	73-5774
8" Left	73-5780	73-5781	73-5782	73-5783	73-5784

	XS	S	M	L	XL
6" Right	73-5750	73-5751	73-5752	73-5753	73-5754
6" Left	73-5760	73-5761	73-5762	73-5763	73-5764

SIZING GUIDE

XS	S	M	L	XL
4"-5"	5"-6"	6"-7"	7"-8"	8"-9"

Measurement taken from wrist circumference

APPLICATION INSTRUCTIONS

1. Open wrist splint and lay flat (inside up) on any clean surface.
2. Place hand on splint so that the strap with the loop portion extends through the index finger and the thumb and attaches to hook on top of your hand.
3. Attach remaining straps while slightly stretching the elastic, starting at the wrist and working up the forearm.

CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Remove any metal stays if possible, close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



TELEPHONE

1-800-426-7353, Ext1



FAX

1-603-623-4111



E-MAIL

Service@corflex.com



MAIL

CORFLEX, Inc.
Attn: Customer Service
669 East Industrial Park Dr
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.