



CORFLEX.
.....

**CORFLEX IS A GLOBALLY BASED
MANUFACTURER AND DISTRIBUTOR OF QUALITY
ORTHOPEDIC AND REHABILITATION PRODUCTS**



WRIST SPLINTS

SIGNATURE FOAM FLANNEL WRIST SPLINT

- > Provides support of the wrist while preserving finger mobility
- > Indicated for Carpal Tunnel Syndrome, tendonitis, wrist strains or sprains, and post-cast removal
- > Constructed of cotton with flannel lining; malleable aluminum palmer stay, flexible dorsal stay and contact closure



PRODUCT NUMBER

	XS	S	M	L	XL
8" Right	73-5920	73-5921	73-5922	73-5923	73-5924
8" Left	73-5930	73-5931	73-5932	73-5933	73-5934
6" Right	73-5720	73-5721	73-5722	73-5723	73-5724
6" Left	73-5730	73-5731	73-5732	73-5733	73-5734
	4XS	3XS	3XS		
5" Right	73-5687	73-5688		6" Right	73-5718
5" Left	73-5697	73-5698		6" Left	73-5728

Measurement taken from wrist circumference

SIZING GUIDE

4XS	3XS	XS	S	M	L	XL
2 1/2"-3 1/2"	3"-4"	4"-5"	5"-6"	6"-7"	7"-8"	8"-9"

APPLICATION INSTRUCTIONS

1. Release all hook and loop closures.
2. Slide hand into splint so wide stay fits comfortably in palm (Note: stay may be shaped to match curvature of wrist).
3. Close all hook and loop closures to desired tension for comfortable fit.

CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Remove any metal stays if possible, close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



TELEPHONE

1-800-426-7353, Ext1



FAX

1-603-623-4111



E-MAIL

Service@corflex.com



MAIL

CORFLEX, Inc.
Attn: Customer Service
669 East Industrial Park Dr
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.