



CORFLEX

**CORFLEX IS A GLOBALLY BASED
MANUFACTURER AND DISTRIBUTOR OF QUALITY
ORTHOPEDIC AND REHABILITATION PRODUCTS**



WRIST SPLINTS

SUEDE WRIST LACER SPLINT

- > Features single-pull lace closure for easy one-hand application
- > Indicated for Carpal Tunnel Syndrome, tendonitis, wrist strains or sprains, and post-cast removal
- > Manufactured from perforated suede-like material with malleable aluminum palmer and dorsal stay



PRODUCT NUMBER

| | XS | S | M | L | XL |
|----------|---------|---------|---------|---------|---------|
| 8" Right | 73-7550 | 73-7551 | 73-7552 | 73-7553 | 73-7554 |
| 8" Left | 73-7560 | 73-7561 | 73-7562 | 73-7563 | 73-7564 |

SIZING GUIDE

| XS | S | M | L | XL |
|-------|-------|-------|-------|-------|
| 4"-5" | 5"-6" | 6"-7" | 7"-8" | 8"-9" |

Measurement taken from wrist circumference

APPLICATION INSTRUCTIONS

1. Open straps.
2. Slide affected hand into wrist splint.
3. Attach strap through thumb and index finger.
4. Grasp hook attached to laces, wrap around and wrist and attach to loop portion.
5. Trim excess portion of strap if desired.

CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Remove any metal stays if possible, close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



TELEPHONE

1-800-426-7353, Ext1



FAX

1-603-623-4111



E-MAIL

Service@corflex.com



MAIL

CORFLEX, Inc.
Attn: Customer Service
669 East Industrial Park Dr
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.