



CORFLEX
.....

**CORFLEX IS A GLOBALLY BASED
MANUFACTURER AND DISTRIBUTOR OF QUALITY
ORTHOPEDIC AND REHABILITATION PRODUCTS**



ANKLE SUPPORTS

ANKLE STIRRUP

- > Universal design conforms to either ankle to provide protection and prevention of ankle inversion or eversion
- > Indicated for chronic ankle instability, ankle sprains or strains and for athletic protection
- > Constructed of semi-rigid shells connected with soft adjustable foot-piece



PRODUCT NUMBER

UNIV

Air - Black	75-1015
Air / Gel - Black	75-1025
Memory Foam - Black	75-1030
Air - White	75-1115
Air / Gel - White	75-1125

SIZING GUIDE

UNIV

Fits up to 14" ankle circumference

Measurement is circumference from base of heel around ankle

APPLICATION INSTRUCTIONS

1. To ensure support and function, stirrup brace must be worn with socks and shoes.
2. The Marathon Ankle Stirrup is a universal device for use on the right or left ankle. Locate medial sticker and place that stirrup shell on inside of affected ankle. Place pad against sock with footpiece under heel.
3. Adjust heel width to give a snug fit by moving adjustable foot piece on each stirrup.
4. While holding shells against ankle, wrap the lower then the upper strap and fasten with hook and loop closures.
5. Once both straps are fastened, re-adjust the lower then upper strap to ensure a proper fit. Fit should be snug, but not so tight it pinches or affects circulation. Apply shoe or sneaker.

CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



TELEPHONE

1-800-426-7353, Ext1



FAX

1-603-623-4111



E-MAIL

Service@corflex.com



MAIL

CORFLEX, Inc.
Attn: Customer Service
669 East Industrial Park Dr
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.