



CORFLEX

**CORFLEX IS A GLOBALLY BASED
MANUFACTURER AND DISTRIBUTOR OF QUALITY
ORTHOPEDIC AND REHABILITATION PRODUCTS**



ANKLE SUPPORTS

LACE-UP ANKLE

- > Features lace-up design, figure-8 support straps and removable medial/lateral stays
- > Indicated for chronic ankle instability, sprains or strains and protection against inversion or eversion ankle injuries
- > Constructed from durable black tricot with brushed tricot lining and removable plastic stays



PRODUCT NUMBER

2XS	XS	S	M	L	XL
75-8000	75-8001	75-8002	75-8003	75-8004	75-8005

SIZING GUIDE

2XS	XS	S	M	L	XL
9"-10"	10"-11"	11"-12"	12"-13"	13"-14"	14"-15"

Measurement is circumference from base of heel around ankle

APPLICATION INSTRUCTIONS

1. Unfasten all straps, loosen laces and lift up tongue of support.
2. Grasp support on either side of laces, slip foot through support, making sure heel is positioned comfortably in opening.
3. With ankle positioned at 90° snug laces from distal to proximal, tighten and tie to desired comfort level.
4. Grasp "figure 8" support straps on each side of lacing. Cross straps in front of foot, wrap under foot and up opposite side of ankle, securing hook portion to loop on side of support. Trim support straps if necessary.
5. Grasp cinch straps on each side of support and bring down over "figure 8" support straps. Ankle support should be snug, but not inhibiting circulation.

NOTE: Optional plastic stays are located on each side under cinch straps and may be removed if desired.

CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Remove any metal stays if possible, close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



TELEPHONE

1-800-426-7353, Ext1



FAX

1-603-623-4111



E-MAIL

Service@corflex.com



MAIL

CORFLEX, Inc.
Attn: Customer Service
669 East Industrial Park Dr
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.