



**CORFLEX**

**CORFLEX IS A GLOBALLY BASED  
MANUFACTURER AND DISTRIBUTOR OF QUALITY  
ORTHOPEDIC AND REHABILITATION PRODUCTS**



## BACK SUPPORTS

### E/N LUMBAR SUPPORT

- > Features tapered elastic side panels with neoprene posterior back panel and pocket that accepts either a thermoplastic insert, gel pack, or pressure pad (sold separately); also features double-pull tension straps for added compression
- > Indicated for moderate to severe low back pain, lumbar muscle weakness, lumbar strain or sprain, and lumbar instability
- > Manufactured from premium quality elastic with contact closure



Insert options:  
37-6031 Thermoplastic Insert  
77-9999 Pressure Pad  
72-7006 Gel Pack

### PRODUCT NUMBER

	XS	S	M	L	XL
Beige	77-6220	77-6221	77-6222	77-6223	77-6224
Black	77-6120	77-6121	77-6122	77-6123	77-6124

	2XL	3XL	4XL
Beige	77-6225	77-6226	77-6227
Black	77-6125	77-6126	77-6127

### SIZING GUIDE

XS	S	M	L	XL
18"-24"	24"-30"	30"-36"	36"-42"	42"-48"

2XL	3XL	4XL
48"-52"	52"-56"	56"-60"

Measurement taken from hip circumference

### APPLICATION INSTRUCTIONS

1. Lay back support flat, locate top of support by opening pocket (top).
2. If applicable, place gel pack, thermoplastic insert, or pressure pad into pocket if desired.
3. Grasp back support and center on small of back.
4. Pull left side firmly across middle section, repeat for right side. If applicable, attach double pull onto Velcro closure.
5. If product comes with suspenders, raise suspenders to shoulders and adjust to your height by sliding the buckle (suspenders are removable).

### CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Remove any metal stays if possible, close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

### HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



**TELEPHONE**

1-800-426-7353, Ext1



**FAX**

1-603-623-4111



**E-MAIL**

Service@corflex.com



**MAIL**

CORFLEX, Inc.  
Attn: Customer Service  
669 East Industrial Park Dr  
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109  
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.