



CORFLEX

**CORFLEX IS A GLOBALLY BASED
MANUFACTURER AND DISTRIBUTOR OF QUALITY
ORTHOPEDIC AND REHABILITATION PRODUCTS**



BACK SUPPORTS

INDUSTRIAL BACK SUPPORT

- > Criss-cross design limits stress on the back and encourages proper lifting techniques; suspenders optional
- > Indicated for lumbar muscle weakness, lumbar strains or sprains, moderate low back pain or prevention of back strain
- > Manufactured from premium quality elastic with contact closure



PRODUCT NUMBER

	XS	S	M	L	XL	2XL
Straps	77-8140	77-8141	77-8142	77-8143	77-8144	77-8145
No Straps	77-9150	77-9151	77-9152	77-9153	77-9154	77-9155
	3XL	4XL				
Straps	77-8146	77-8147				
No Straps	77-9156	77-9157	Note: Also available in Beige and White			

SIZING GUIDE

	XS	S	M	L	XL	2XL
	18"-24"	24"-30"	30"-36"	36"-42"	42"-48"	48"-52"
	3XL	4XL				
	52"-56"	56"-60"				

Measurement taken from hip circumference

APPLICATION INSTRUCTIONS

1. Lay back support flat, locate top of support.
2. Grasp back support and center in small of back. Support should rest over waist line at the top of your pants.
3. If applicable, raise suspenders to shoulders and adjust to your height by sliding the buckle-(suspenders are removable).
4. Detach left and right 4" double pull straps. Using left hand, attach left strap to patch on right side of belt. Finish by overlapping the double pull to the left side of belt and attaching to patch.

CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Remove any metal stays if possible, close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



TELEPHONE

1-800-426-7353, Ext1



FAX

1-603-623-4111



E-MAIL

Service@corflex.com



MAIL

CORFLEX, Inc.
Attn: Customer Service
669 East Industrial Park Dr
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.