



**CORFLEX**  
.....

**CORFLEX IS A GLOBALLY BASED  
MANUFACTURER AND DISTRIBUTOR OF QUALITY  
ORTHOPEDIC AND REHABILITATION PRODUCTS**



## BACK SUPPORTS

### LOW PROFILE INDUSTRIAL BACK SUPPORT

- > Comfortable low profile design limits stress on back, encourages proper lifting techniques; suspenders optional
- > Indicated for lumbar muscle weakness, lumbar strains or sprains, moderate low back pain or prevention of back strain
- > Manufactured from premium quality elastic w/contact closure



#### PRODUCT NUMBER

	XS	S	M	L	XL	2XL
Straps	77-8840	77-8841	77-8842	77-8843	77-8844	77-8845
No Straps	77-9840	77-9841	77-9842	77-9843	77-9844	77-9845
	3XL	4XL	Note: Also available in Beige and White			
Straps	77-8846	77-8847				
No Straps	77-9846	77-9847				

#### SIZING GUIDE

	XS	S	M	L	XL	2XL
	18"-24"	24"-30"	30"-36"	36"-42"	42"-48"	48"-52"
	3XL	4XL	Measurement taken from hip circumference			
	52"-56"	56"-60"				

### APPLICATION INSTRUCTIONS

1. Lay back support flat, locate top of support.
2. Grasp back support and center in small of back. Support should rest over waist line at the top of your pants.
3. If applicable, raise suspenders to shoulders and adjust to your height by sliding the buckle-(suspenders are removable).
4. Detach left and right 4" double pull straps. Using left hand, attach left strap to patch on right side of belt. Finish by overlapping the double pull to the left side of belt and attaching to patch.

### CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Remove any metal stays if possible, close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

### HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



#### TELEPHONE

1-800-426-7353, Ext1



#### FAX

1-603-623-4111



#### E-MAIL

Service@corflex.com



#### MAIL

CORFLEX, Inc.  
Attn: Customer Service  
669 East Industrial Park Dr  
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109  
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.