



CORFLEX

**CORFLEX IS A GLOBALLY BASED
MANUFACTURER AND DISTRIBUTOR OF QUALITY
ORTHOPEDIC AND REHABILITATION PRODUCTS**



ANKLE IMMOBILIZERS

R.O.M. METAL WALKER

- > Adjustable hinge limits range of motion during healing process
- > Indicated for stable foot/ankle fractures, severe sprains or strains, soft tissue injuries, post-op and post-cast removal
- > Constructed of durable malleable metal uprights, easy to set hinge, padded liner, contact closure straps



PRODUCT NUMBER

S	M	L
79-1070	79-1071	79-1072

SIZING GUIDE

	MENS SIZE	WOMENS SIZE
Small	5 - 6	5 - 7
Medium	6 - 10	7 - 11
Large	10 - 12	11 - 13 1/2

Measured by shoe size

APPLICATION INSTRUCTIONS

1. Loosen straps, remove liner from boot and set pads and wrench aside.
2. Place foot in liner (foot should be flat on the ground with ankle at 90 degree angle). Make sure heel fits snugly into posterior portion of the liner.
3. Place pads over malleolus and secure to inside of liner with hook dots.
4. Secure liner to lower leg with contact closures starting at the bottom and working up the leg.
5. Spread uprights apart and step into boot, making sure to align the uprights with the medial/lateral side of liner secured to lower leg.
6. Secure boot straps starting at bottom and working up the leg.
7. Set hinge to allow for or limit range of motion by sliding cover open and placing pin at appropriate degree of plantar or dorsiflexion.

Note: (Each side should be set with identical settings). Wrench may be used to unlock (turn counterclockwise) or lock (turn clockwise) hinge as needed.

CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Wipe plastic shell with damp cloth, you may use disinfectant or mild soap, and wipe dry.

For products with a removable liner, you may hand wash liner in warm water with mild soap, air dry, do not use high temperatures to wash or dry.

HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



TELEPHONE

1-800-426-7353, Ext1



FAX

1-603-623-4111



E-MAIL

Service@corflex.com



MAIL

CORFLEX, Inc.
Attn: Customer Service
669 East Industrial Park Dr
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.