



CORFLEX

**CORFLEX IS A GLOBALLY BASED
MANUFACTURER AND DISTRIBUTOR OF QUALITY
ORTHOPEDIC AND REHABILITATION PRODUCTS**



ANKLE IMMOBILIZERS

PNEUMATIC METAL WALKER

- > Air pump provides desired compression level and accommodates for swelling
- > Indicated for stable foot/ankle fractures, severe sprains or strains, soft tissue injuries, post-op and post-cast removal
- > Constructed of durable malleable metal uprights, circumferential bladder, contact closure straps



PRODUCT NUMBER

	S	M	L	XL
Tall	79-1080	79-1081	79-1082	79-1083
Short	79-1090	79-1091	79-1092	79-1093

SIZING GUIDE

	MENS SIZE	WOMENS SIZE
Small	4 - 6	5 - 7
Medium	6 - 10	7 - 11
Large	10 - 12	11 - 13
X-Large	12 +	13 +

Measured by shoe size

APPLICATION INSTRUCTIONS

1. Application instructions see page 2.

CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Wipe plastic shell with damp cloth, you may use disinfectant or mild soap, and wipe dry.

For products with a removable liner, you may hand wash liner in warm water with mild soap, air dry, do not use high temperatures to wash or dry.

HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



TELEPHONE

1-800-426-7353, Ext1



FAX

1-603-623-4111



E-MAIL

Service@corflex.com



MAIL

CORFLEX, Inc.
Attn: Customer Service
669 East Industrial Park Dr
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.

AIR WALKER INSTRUCTIONS

A

For first time application, loosen straps and remove liner from boot.



B

Place foot in liner and secure with contact closure. Make sure the heel fits snugly into the posterior portion of the liner. Fasten the foot flaps on the liner first, making sure they are snug. Wrap and fasten the leg portion of the liner, again making sure that it is snug from bottom to top. Some trimming of the liner may be necessary to accomplish optimum fit.



C

Spread the uprights using both hands and step into boot, aligning uprights with midline of the ankle.

Note: If uprights have plastic tube liner, remove them now to affix upright to liner.



D

Secure boot straps, starting at the toes and working up the leg.



To **INFLATE** air chambers simply insert the **WHITE** tip of the air pump and squeeze pump. **DO NOT OVER INFLATE**. The air chamber should only be inflated to a comfortable level. To **DEFLATE** insert **BLACK** tip into the valve and squeeze pump until comfortable level is attained

Slight air loss may be experienced during periods of wear and air can easily be added.

Foam pads are provided to alleviate the problem of excess rubbing and pads can be placed either inside the liner or between the liner and the upright stay.

