



CORFLEX

**CORFLEX IS A GLOBALLY BASED
MANUFACTURER AND DISTRIBUTOR OF QUALITY
ORTHOPEDIC AND REHABILITATION PRODUCTS**



CERVICAL SUPPORTS

MEDIC-AIR CERVICAL SLEEP PILLO

- > Positions between pillow and pillowcase to provide the gentle and adjustable support of air for the cervical spine; easily inflates to desired comfort level and deflates for portability
- > Provides cervical support while sleeping
- > Manufactured from medical grade durable nylon-coated vinyl with open-valve system



PRODUCT NUMBER

	SINGLE	CASE/12
Blue	84-6051-100	84-6051-300

SIZING GUIDE

Product measures 21" x 4"

APPLICATION INSTRUCTIONS

1. Unfold Pillo completely.
2. Gently & moderately blow air into Pillo with 3 or 4 breaths. (The Pillo should be soft rather than firm to gain the benefits of "floating air" action.)
3. Place cap firmly into valve (moisten slightly if it tends to stick.)
4. Hold Pillo on lap and with hand, press down in middle, forcing valve up. With the other hand, press on valve until it recedes into Pillo.

CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Wipe exterior surface with damp cloth. You may use disinfectant or mild soap. Never use cleaning fluid, let pillow air dry, no not use any appliance for drying.

HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



TELEPHONE

1-800-426-7353, Ext1



FAX

1-603-623-4111



E-MAIL

Service@corflex.com



MAIL

CORFLEX, Inc.
Attn: Customer Service
669 East Industrial Park Dr
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.