



CORFLEX

**CORFLEX IS A GLOBALLY BASED
MANUFACTURER AND DISTRIBUTOR OF QUALITY
ORTHOPEDIC AND REHABILITATION PRODUCTS**



HINGED KNEE SUPPORTS / NEOPRENE

CoolTex Anterior Closure Knee Wrap W/Hinge

- > Features wrap-around design for ease of application with heavy-duty hinges for optimal medial/lateral stability
- > Indicated for medial/lateral instability, muscle/ligament weakness or injury, and degenerative joint disease; ideal alternative for patients with sensitivity to neoprene
- > Manufactured from CoolTex, a lightweight breathable neoprene alternative, 100% latex free and hypo-allergenic



PRODUCT NUMBER

| | S | M | L | XL | 2XL | 3XL | 4XL |
|------------|---------|---------|---------|---------|---------|---------|---------|
| 13" Op Pop | 86-1303 | 86-1304 | 86-1305 | 86-1306 | 86-1307 | 86-1308 | 86-1309 |

SIZING GUIDE

| | 6" ABOVE MID-PATELLA | MID-PATELLA | 6" BELOW MID-PATELLA |
|----------|----------------------|-------------|----------------------|
| Small | 16"-17" | 12"-14" | 11"-12" |
| Medium | 18"-19" | 14"-16" | 13"-14" |
| Large | 19"-21" | 16"-18" | 15"-16" |
| X-Large | 21"-23" | 18"-20" | 17"-18" |
| 2X-Large | 23"-25" | 20"-22" | 19"-20" |
| 3X-Large | 25"-27" | 22"-24" | 21"-22" |
| 4X-Large | 27"-28" | 24"-26" | 22"-23" |

APPLICATION INSTRUCTIONS

1. Open knee wrap and all straps.
2. Position wrap with gray on inside and line up condyle pads slightly above knee.
3. Fasten support straps beginning with most distal to desired comfort level.
4. Fasten remaining support straps.

CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Remove any metal stays if possible, close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



TELEPHONE

1-800-426-7353, Ext1



FAX

1-603-623-4111



E-MAIL

Service@corflex.com



MAIL

CORFLEX, Inc.
Attn: Customer Service
669 East Industrial Park Dr
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.