



**CORFLEX**

**CORFLEX IS A GLOBALLY BASED  
MANUFACTURER AND DISTRIBUTOR OF QUALITY  
ORTHOPEDIC AND REHABILITATION PRODUCTS**



**KNEE SLEEVES / NEOPRENE**

**KNEE SLEEVE**

- > Provides warmth, support and compression for the knee
- > Indicated for moderate knee strains/sprains, knee injuries and arthritis
- > Manufactured from premium grade latex-free neoprene



Also available in 3X & 4X

**PRODUCT NUMBER**

1/8"	XS	S	M	L	XL	2XL
13"	88-0012	88-0013	88-0014	88-0015	88-0016	88-0017
Op Pop	88-0022	88-0023	88-0024	88-0025	88-0026	88-0027
Op Pat	88-0032	88-0033	88-0034	88-0035	88-0036	88-0037
Op Pat/Pop	88-0042	88-0043	88-0044	88-0045	88-0046	88-0047
3/16"	XS	S	M	L	XL	2XL
13"	88-0092	88-0093	88-0094	88-0095	88-0096	88-0097
Op Pop	88-0102	88-0103	88-0104	88-0105	88-0106	88-0107
Op Pat	88-0112	88-0113	88-0114	88-0115	88-0116	88-0117
Op Pat/Pop	88-0122	88-0123	88-0124	88-0125	88-0126	88-0127

**SIZING GUIDE**

	6" ABOVE MID-PATELLA	MID-PATELLA	6" BELOW MID-PATELLA
X-Small	16"-18"	12"-13"	11"-12"
Small	17"-19"	13"-14"	12"-13"
Medium	18"-20"	14"-15"	13"-14"
Large	19"-21"	15"-16"	14"-15"
X-Large	20"-22"	16"-18"	15"-16"
2X-Large	21"-23"	18"-20"	16"-17"
3X-Large	24"-26"	20"-22"	17"-18"
4X-Large	27"-29"	22"-24"	18"-19"

**APPLICATION INSTRUCTIONS**

1. With black outside exposed insert foot into the largest end of support and slide over knee.
2. Make sure seam is placed posteriorly, in back of leg.
3. Your kneecap (patella) should be centered in one opening and the other opening should be centered in back of leg (popliteal).
4. Adjust placement as required.

**CARE & CLEANING**

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

**HOW TO PLACE AN ORDER**

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



**TELEPHONE**

1-800-426-7353, Ext1



**FAX**

1-603-623-4111



**E-MAIL**

Service@corflex.com



**MAIL**

CORFLEX, Inc.  
Attn: Customer Service  
669 East Industrial Park Dr  
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109  
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.