



CORFLEX

**CORFLEX IS A GLOBALLY BASED
MANUFACTURER AND DISTRIBUTOR OF QUALITY
ORTHOPEDIC AND REHABILITATION PRODUCTS**



KNEE SLEEVES / NEOPRENE

KNEE SLEEVE W/ANTERIOR PAD

- > Provides warmth, protection and compression for the knee, also features anterior pad over patella region for added protection
- > Indicated for moderate knee sprains/strains, knee injuries and arthritis
- > Manufactured from premium grade latex-free neoprene



Also available in 3X & 4X

PRODUCT NUMBER

1/8"	XS	S	M	L	XL	2XL
13"	88-0212	88-0213	88-0214	88-0215	88-0216	88-0217
Op Pop	88-0242	88-0243	88-0244	88-0245	88-0246	88-0247
Op Pat	88-0232	88-0233	88-0234	88-0235	88-0236	88-0237
Op Pat/Pop	88-0222	88-0223	88-0224	88-0225	88-0226	88-0227
3/8"	XS	S	M	L	XL	2XL
13"	88-0172	88-0173	88-0174	88-0175	88-0176	88-0177
Op Pop	88-0202	88-0203	88-0204	88-0205	88-0206	88-0207
Op Pat	88-0192	88-0193	88-0194	88-0195	88-0196	88-0197
Op Pat/Pop	88-0182	88-0183	88-0184	88-0185	88-0186	88-0187

SIZING GUIDE

	6" ABOVE MID-PATELLA	MID-PATELLA	6" BELOW MID-PATELLA
X-Small	16"-18"	12"-13"	11"-12"
Small	17"-19"	13"-14"	12"-13"
Medium	18"-20"	14"-15"	13"-14"
Large	19"-21"	15"-16"	14"-15"
X-Large	20"-22"	16"-18"	15"-16"
2X-Large	21"-23"	18"-20"	16"-17"
3X-Large	24"-26"	20"-22"	17"-18"
4X-Large	27"-29"	22"-24"	18"-19"

APPLICATION INSTRUCTIONS

1. With black outside exposed insert foot into the largest end of support and slide over knee.
2. Make sure seam is placed posteriorly, in back of leg.
3. Your kneecap (patella) should be centered in one opening and the other opening should be centered in back of leg (popliteal).
4. Adjust placement as required.

CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



TELEPHONE

1-800-426-7353, Ext1



FAX

1-603-623-4111



E-MAIL

Service@corflex.com



MAIL

CORFLEX, Inc.
Attn: Customer Service
669 East Industrial Park Dr
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.