



**CORFLEX.**  
.....

**CORFLEX IS A GLOBALLY BASED  
MANUFACTURER AND DISTRIBUTOR OF QUALITY  
ORTHOPEDIC AND REHABILITATION PRODUCTS**



**HINGED KNEE SUPPORTS / NEOPRENE**

**KNEE SLEEVE W/HINGE**

- > Features sleeve design with heavy-duty hinges for optimal medial/lateral support
- > Indicated for medial/lateral instability, muscle/ligament weakness or injury, and degenerative joint disease
- > Manufactured from premium grade latex-free neoprene



**PRODUCT NUMBER**

1/8"	XS	S	M	L	XL
13"	88-0302	88-0303	88-0304	88-0305	88-0306
13" Op Pop	88-0312	88-0313	88-0314	88-0315	88-0316

1/8"	2XL	3XL	4XL
13"	88-0307	88-0308	88-0309
13" Op Pop	88-0317	88-0318	88-0319

3/16"	XS	S	M	L	XL
13"	88-0322	88-0323	88-0324	88-0325	88-0326
13" Op Pop	88-0332	88-0333	88-0334	88-0335	88-0336

3/16"	2XL	3XL	4XL
13"	88-0327	88-0328	88-0329
13" Op Pop	88-0337	88-0338	88-0339

**SIZING GUIDE**

	6" ABOVE MID-PATELLA	MID-PATELLA	6" BELOW MID-PATELLA
X-Small	16"-18"	12"-13"	11"-12"
Small	17"-19"	13"-14"	12"-13"
Medium	18"-20"	14"-15"	13"-14"
Large	19"-21"	15"-16"	14"-15"
X-Large	20"-22"	16"-18"	15"-16"
2X-Large	21"-23"	18"-20"	16"-17"
3X-Large	24"-26"	20"-22"	17"-18"
4X-Large	27"-29"	22"-24"	18"-19"

**APPLICATION INSTRUCTIONS**

1. Unfasten and loosen the straps.
2. Grasp the edge of the wider portion of the sleeve, keeping the seam positioned to posterior of leg.
3. Insert foot into sleeve and pull onto leg.
4. Position the sleeve so that stays or hinges are parallel to each other on the side of leg, and the top of the front cut out aligns with

**CARE & CLEANING**

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Remove any metal stays if possible, close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

669 East Industrial Park Drive - Manchester, NH 03109  
Phone (603) 623-3344 - Fax (603) 623-4111 - [www.corflex.com](http://www.corflex.com)

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.



**CORFLEX.**  
.....

**CORFLEX IS A GLOBALLY BASED  
MANUFACTURER AND DISTRIBUTOR OF QUALITY  
ORTHOPEDIC AND REHABILITATION PRODUCTS**



**HINGED KNEE SUPPORTS / NEOPRENE**

the top of the kneecap (patella) and rear cut out is centered in the back of the knee (popliteal) as required.

5. Sleeve should sit on leg with approximately 6" of material above mid kneecap (patella).

6. Fasten the support straps at the thigh and calf as required for comfortable fit without inhibiting circulation.

**HOW TO PLACE AN ORDER**

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



**TELEPHONE**

1-800-426-7353, Ext1



**FAX**

1-603-623-4111



**E-MAIL**

Service@corflex.com



**MAIL**

CORFLEX, Inc.  
Attn: Customer Service  
669 East Industrial Park Dr  
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109  
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.