



CORFLEX.

**CORFLEX IS A GLOBALLY BASED
MANUFACTURER AND DISTRIBUTOR OF QUALITY
ORTHOPEDIC AND REHABILITATION PRODUCTS**



HINGED KNEE SUPPORTS / NEOPRENE

KNEE SLEEVE W/R.O.M. HINGE

- > Features R.O.M. hinges that allow for full flexion and extension control and sleeve design
- > Indicated for medial/lateral instability, muscle/ligament weakness or injury, and degenerative joint disease where extension/flexion control is required
- > Manufactured from premium grade latex-free neoprene



PRODUCT NUMBER

3/16"	XS	S	M	L	XL
13"	88-0362	88-0363	88-0364	88-0365	88-0366
13" Op Pop	88-0372	88-0373	88-0374	88-0375	88-0376

3/16"	2XL	3XL	4XL
13"	88-0367	88-0368	88-0369
13" Op Pop	88-0377	88-0378	88-0379

SIZING GUIDE

	6" ABOVE MID-PATELLA	MID-PATELLA	6" BELOW MID-PATELLA
X-Small	16"-18"	12"-13"	11"-12"
Small	17"-19"	13"-14"	12"-13"
Medium	18"-20"	14"-15"	13"-14"
Large	19"-21"	15"-16"	14"-15"
X-Large	20"-22"	16"-18"	15"-16"
2X-Large	21"-23"	18"-20"	16"-17"
3X-Large	24"-26"	20"-22"	17"-18"
4X-Large	27"-29"	22"-24"	18"-19"

APPLICATION INSTRUCTIONS

1. Unfasten and loosen the straps.
2. Grasp the edge of the wider portion of the sleeve, keeping the seam positioned to posterior of leg.
3. Insert foot into sleeve and pull onto leg.
4. Position the sleeve so that stays or hinges are parallel to each other on the side of leg, and the top of the front cut out aligns with the top of the kneecap (patella) and rear cut out is centered in the back of the knee (popliteal) as required.
5. Sleeve should sit on leg with approximately 6" of material above mid kneecap (patella).
6. Fasten the support straps at the thigh and calf as required for comfortable fit without inhibiting circulation.

CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Remove any metal stays if possible, close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



TELEPHONE

1-800-426-7353, Ext1



FAX

1-603-623-4111



E-MAIL

Service@corflex.com



MAIL

CORFLEX, Inc.
Attn: Customer Service
669 East Industrial Park Dr
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.