



CORFLEX

**CORFLEX IS A GLOBALLY BASED
MANUFACTURER AND DISTRIBUTOR OF QUALITY
ORTHOPEDIC AND REHABILITATION PRODUCTS**



KNEE WRAPS / NEOPRENE

UNIVERSAL KNEE WRAP

- > Provides warmth, support, and compression for the knee, universal design allows for inventory reduction; available with spiral stays or hinges
- > Indicated for moderate knee strains/sprains, knee injuries and arthritis
- > Manufactured from premium grade latex-free soft loop neoprene



PRODUCT NUMBER

1 1/2"	UNIV
1/8" Ultra	88-1000
1/8" Ultra W/Stays	88-1001
1/8" Wrap	88-1010
1/8" Wrap W/Stays	88-1020
1/8" Wrap W/Hinge	88-1030

SIZING GUIDE

	ULTRA UNIV	WRAP UNIV
Fits up to	25"	22"

Measurement taken 6" above mid-patella

APPLICATION INSTRUCTIONS

1. Position patella opening about 1" above the center of the patella (knee cap).
2. Tighten thigh strap and pull support down so opening spreads around patella and tighten the calf strap. This technique will eliminate bunching of the material and provide an excellent fit.
3. Straps may be re-adjusted for desired comfort level.

CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Remove any metal stays if possible, close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



TELEPHONE

1-800-426-7353, Ext1



FAX

1-603-623-4111



E-MAIL

Service@corflex.com



MAIL

CORFLEX, Inc.
Attn: Customer Service
669 East Industrial Park Dr
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.