



CORFLEX

**CORFLEX IS A GLOBALLY BASED
MANUFACTURER AND DISTRIBUTOR OF QUALITY
ORTHOPEDIC AND REHABILITATION PRODUCTS**



HINGED KNEE SUPPORTS / NEOPRENE

ANTERIOR CLOSURE KNEE WRAP W/HINGE

- > Features wrap-around design for ease of application with heavy-duty hinges for optimal medial /lateral support; available in two lengths
- > Indicated for medial/lateral instability, muscle/ligament weakness or injury, and degenerative joint disease
- > Manufactured from premium grade latex-free neoprene



Also available in 3X & 4X

PRODUCT NUMBER

3/16"	S	M	L	XL	2XL
13"	88-1313	88-1314	88-1315	88-1316	88-1317
13" Op Pop	88-1303	88-1304	88-1305	88-1306	88-1307
16"	88-1673	88-1674	88-1675	88-1676	88-1677
16" Op Pop	88-1663	88-1664	88-1665	88-1666	88-1667

1/8"	S	M	L	XL	2XL
13"	88-1213	88-1214	88-1215	88-1216	88-1217
13" Op Pop	88-1203	88-1204	88-1205	88-1206	88-1207
13" Perf	87-1213	87-1214	87-1215	87-1216	87-1217
13" Perf OP	87-1203	87-1204	87-1205	87-1206	87-1207
16" Perf	87-1273	87-1274	87-1275	87-1276	87-1277
16" Perf OP	87-1263	87-1264	87-1265	87-1266	87-1267

SIZING GUIDE

	6" ABOVE MID-PATELLA	MID-PATELLA	6" BELOW MID-PATELLA
Small	16"-17"	12"-14"	11"-12"
Medium	18"-19"	14"-16"	13"-14"
Large	19"-21"	16"-18"	15"-16"
X-Large	21"-23"	18"-20"	17"-18"
2X-Large	23"-25"	20"-22"	19"-20"
3X-Large	25"-27"	22"-24"	21"-22"
4X-Large	27"-28"	24"-26"	22"-23"

APPLICATION INSTRUCTIONS

1. Open knee wrap and all straps.
2. Position wrap with gray on inside and line up condyle pads slightly above knee.
3. Fasten support straps beginning with most distal to desired comfort level.
4. Fasten remaining support straps.

CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Remove any metal stays if possible, close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



TELEPHONE

1-800-426-7353, Ext1



FAX

1-603-623-4111



E-MAIL

Service@corflex.com



MAIL

CORFLEX, Inc.
Attn: Customer Service
669 East Industrial Park Dr

669 East Industrial Park Drive - Manchester, NH 03109
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.



CORFLEX.
.....

**CORFLEX IS A GLOBALLY BASED
MANUFACTURER AND DISTRIBUTOR OF QUALITY
ORTHOPEDIC AND REHABILITATION PRODUCTS**



HINGED KNEE SUPPORTS / NEOPRENE

Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.