



**CORFLEX**

**CORFLEX IS A GLOBALLY BASED  
MANUFACTURER AND DISTRIBUTOR OF QUALITY  
ORTHOPEDIC AND REHABILITATION PRODUCTS**



**HINGED KNEE SUPPORTS / NEOPRENE**

**ANTERIOR CLOSURE KNEE WRAP W/R.O.M. HINGE**

- > Features R.O.M. hinges that allow for full flexion and extension control and wrap-around design for ease of application; available in two lengths
- > Indicated for medial/lateral instability, muscle/ligament weakness or injury, and degenerative joint disease where extension/flexion control is required
- > Manufactured from premium grade latex-free neoprene



Also available in 3X & 4X

**PRODUCT NUMBER**

3/16"	S	M	L	XL	2XL
13"	88-1353	88-1354	88-1355	88-1356	88-1357
13" Op Pop	88-1343	88-1344	88-1345	88-1346	88-1347
16"	88-1693	88-1694	88-1695	88-1696	88-1697
16" Op Pop	88-1683	88-1684	88-1685	88-1686	88-1687

1/8"	S	M	L	XL	2XL
13" Perf	87-1253	87-1254	87-1255	87-1256	87-1257
13" Perf OP	87-1243	87-1244	87-1245	87-1246	87-1247
16" Perf	87-1293	87-1294	87-1295	87-1296	87-1297
16" Perf OP	87-1283	87-1284	87-1285	87-1286	87-1287

**SIZING GUIDE**

	6" ABOVE MID-PATELLA	MID-PATELLA	6" BELOW MID-PATELLA
Small	16"-17"	12"-14"	11"-12"
Medium	18"-19"	14"-16"	13"-14"
Large	19"-21"	16"-18"	15"-16"
X-Large	21"-23"	18"-20"	17"-18"
2X-Large	23"-25"	20"-22"	19"-20"
3X-Large	25"-27"	22"-24"	21"-22"
4X-Large	27"-28"	24"-26"	22"-23"

**APPLICATION INSTRUCTIONS**

1. Open knee wrap and straps.
2. Position wrap with gray on inside and line up condyle pads slightly above knee.
3. Fasten support straps beginning with most distal until they fit comfortably around the knee without inhibiting circulation.
4. Fasten remaining support straps.

**CARE & CLEANING**

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Remove any metal stays if possible, close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

**HOW TO PLACE AN ORDER**

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



**TELEPHONE**

1-800-426-7353, Ext1



**FAX**

1-603-623-4111



**E-MAIL**

Service@corflex.com



**MAIL**

CORFLEX, Inc.  
Attn: Customer Service  
669 East Industrial Park Dr  
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109  
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.