



**CORFLEX.**

**CORFLEX IS A GLOBALLY BASED  
MANUFACTURER AND DISTRIBUTOR OF QUALITY  
ORTHOPEDIC AND REHABILITATION PRODUCTS**



**KNEE WRAPS / NEOPRENE**

**ANTERIOR CLOSURE KNEE WRAP W/STAYS**

- > Features wrap-around design for ease of application and provides warmth, support and compression for the knee with spiral stays to prevent roll over
- > Indicated for moderate knee strains/sprains, knee injuries and arthritis
- > Manufactured from premium grade latex-free neoprene



**PRODUCT NUMBER**

13"	S	M	L	XL	2XL
3/16" Op Pop	88-1363	88-1364	88-1365	88-1366	88-1367
3/16"	88-1373	88-1374	88-1375	88-1376	88-1377

13"	3X	4X
3/16" Op Pop	88-1368	88-1369
3/16"	88-1378	88-1379

**SIZING GUIDE**

	6" ABOVE MID-PATELLA	MID-PATELLA	6" BELOW MID-PATELLA
Small	16"-17"	12"-14"	11"-12"
Medium	18"-19"	14"-16"	13"-14"
Large	19"-21"	16"-18"	15"-16"
X-Large	21"-23"	18"-20"	17"-18"
2X-Large	23"-25"	20"-22"	19"-20"
3X-Large	25"-27"	22"-24"	21"-22"
4X-Large	27"-28"	24"-26"	22"-23"

**APPLICATION INSTRUCTIONS**

1. Open knee wrap and all straps.
2. Position wrap with gray on inside and line up condyle pads slightly above knee.
3. Fasten support straps beginning with most distal to desired comfort level.
4. Fasten remaining support straps.

**CARE & CLEANING**

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Remove any metal stays if possible, close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

**HOW TO PLACE AN ORDER**

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



**TELEPHONE**

1-800-426-7353, Ext1



**FAX**

1-603-623-4111



**E-MAIL**

Service@corflex.com



**MAIL**

CORFLEX, Inc.  
Attn: Customer Service  
669 East Industrial Park Dr  
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109  
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.