



CORFLEX
.....

**CORFLEX IS A GLOBALLY BASED
MANUFACTURER AND DISTRIBUTOR OF QUALITY
ORTHOPEDIC AND REHABILITATION PRODUCTS**



CALF THIGH SLEEVES / NEOPRENE

CALF SLEEVE

- > Provides warmth, support and compression for the calf
- > Indicated for strains of the calf or gastroc muscles and shin splints
- > Manufactured from premium grade latex-free neoprene



PRODUCT NUMBER

12"	XS	S	M	L	XL
1/8"	88-1402	88-1403	88-1404	88-1405	88-1406
3/16"	88-1412	88-1413	88-1414	88-1415	88-1416

SIZING GUIDE

XS	S	M	L	XL
12"-13"	13"-14"	14"-15"	15"-17"	17"-18"

Measurement taken from widest part of calf

APPLICATION INSTRUCTIONS

1. With black outside exposed insert foot into the largest end of support and slide over calf.
2. Make sure seam is placed posteriorly, in back of leg.
3. Sleeve should sit below patella (kneecap).
4. Adjust placement as required.

CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



TELEPHONE
1-800-426-7353, Ext1



FAX
1-603-623-4111



E-MAIL
Service@corflex.com



MAIL
CORFLEX, Inc.
Attn: Customer Service
669 East Industrial Park Dr
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.