



CORFLEX

**CORFLEX IS A GLOBALLY BASED
MANUFACTURER AND DISTRIBUTOR OF QUALITY
ORTHOPEDIC AND REHABILITATION PRODUCTS**



BACK SUPPORTS

TARGET BACK WRAP

- > Provides support, compression and warmth for the back, with sewn-in tapered pressure pad; universal design allows for inventory reduction
- > Indicated for mild low back pain, lumbar sprains/strains or weakness
- > Manufactured from premium latex-free neoprene



PRODUCT NUMBER

UNV	2XL/3XL
88-1800	88-1801

SIZING GUIDE

UNV	2XL/3XL
24"-46"	46"-56"

Measurement taken from hip circumference

APPLICATION INSTRUCTIONS

1. Release hook and loop closure.
2. Grasp back support and center on small of back, making sure widest part of contoured pressure pad is at top.
3. Pull ends of support toward front of body and secure hook and loop.
4. Grasp side pull straps, pull firmly and attach to the side for desired tension and support.

CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Remove any metal stays if possible, close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



TELEPHONE

1-800-426-7353, Ext1



FAX

1-603-623-4111



E-MAIL

Service@corflex.com



MAIL

CORFLEX, Inc.
Attn: Customer Service
669 East Industrial Park Dr
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.