



**CORFLEX**

**CORFLEX IS A GLOBALLY BASED  
MANUFACTURER AND DISTRIBUTOR OF QUALITY  
ORTHOPEDIC AND REHABILITATION PRODUCTS**



## THUMB SPLINTS

### TARGET SEWN THUMB

- > Designed to provide moderate support, warmth, compression and protection for the thumb, also available with stays for added support and immobilization
- > Indicated for tendonitis, and thumb strains or sprains
- > Manufactured from premium grade latex-free neoprene, optional plastic/metal stays



### PRODUCT NUMBER

<b>UNIV</b>
88-2040
W/Stay
88-2042

### SIZING GUIDE

<b>UNIV</b>
Fits up to 9" wrist circumference

### APPLICATION INSTRUCTIONS

1. Place affected thumb in opening, making sure smaller seam is on inside of thumb.
2. Grasp short portion of wrap and pull around wrist.
3. Grasp longer portion of wrap, pull around opposite side of wrist and attach hook closure.
4. Adjust to desired tension of comfort level.

NOTE: Plastic and metal stays are located in pocket with tab on outside of thumb and one or both may be removed if desired.

### CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Remove any metal stays if possible, close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

### HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



**TELEPHONE**

1-800-426-7353, Ext1



**FAX**

1-603-623-4111



**E-MAIL**

Service@corflex.com



**MAIL**

CORFLEX, Inc.  
Attn: Customer Service  
669 East Industrial Park Dr  
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109  
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.