



CORFLEX
.....

**CORFLEX IS A GLOBALLY BASED
MANUFACTURER AND DISTRIBUTOR OF QUALITY
ORTHOPEDIC AND REHABILITATION PRODUCTS**



WRIST SPLINTS / NEOPRENE

TARGET WRIST STABILIZER

- > Easy to apply wrap-around design provides warmth, compression and protection for the wrist
- > Indicated for Carpal Tunnel Syndrome, tendonitis, and wrist strains or sprains
- > Manufactured from latex-free soft loop neoprene



PRODUCT NUMBER

	UNIV
	88-2050
Perforated	87-2050

SIZING GUIDE

	UNIV
Fits up to 9" wrist circumference	

APPLICATION INSTRUCTIONS

1. Place thumb of affected hand in opening.
2. Grasp hook closure tab, wrap around wrist and attach hook closure onto loop material.
3. Adjust to desired tension of comfort level.

CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Remove any metal stays if possible, close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:

TELEPHONE	FAX	E-MAIL	MAIL
1-800-426-7353, Ext1	1-603-623-4111	Service@corflex.com	CORFLEX, Inc. Attn: Customer Service 669 East Industrial Park Dr Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.