



CORFLEX


**CORFLEX IS A GLOBALLY BASED
MANUFACTURER AND DISTRIBUTOR OF QUALITY
ORTHOPEDIC AND REHABILITATION PRODUCTS**



WRIST SPLINTS / NEOPRENE

TARGET WRIST STABILIZER W/STAY

- > Easy to apply wrap-around design provides support, warmth, compression and protection for the wrist
- > Indicated for Carpal Tunnel Syndrome, tendonitis, and wrist strains or sprains
- > Manufactured from latex-free soft loop neoprene with removable malleable palmer stay

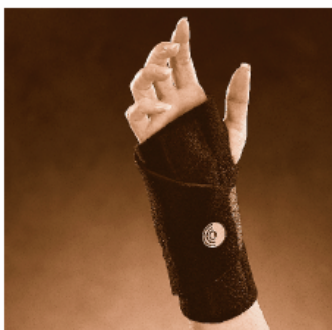

PRODUCT NUMBER

	UNIV NEOPRENE	UNIV PERFORATED
Pedi Right	88-2056	
Pedi Left	88-2057	
Right	88-2060	87-2060
Left	88-2070	87-2070
Extended Right	88-2061	87-2061
Extended Left	88-2071	87-2071

SIZING GUIDE

UNIV
Fits up to 9" wrist circumference

APPLICATION INSTRUCTIONS

1. Place thumb of affect hand in opening, making sure stay is centered on palm of hand.
2. Grasp hook closure tab, wrap around wrist and attach hook closure onto loop material.
3. Adjust to desired tension of comfort level.

CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Remove any metal stays if possible, close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



TELEPHONE

1-800-426-7353, Ext1



FAX

1-603-623-4111



E-MAIL

Service@corflex.com



MAIL

CORFLEX, Inc.
Attn: Customer Service
669 East Industrial Park Dr
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.