



**CORFLEX**

**CORFLEX IS A GLOBALLY BASED  
MANUFACTURER AND DISTRIBUTOR OF QUALITY  
ORTHOPEDIC AND REHABILITATION PRODUCTS**



## TENNIS ELBOW SUPPORTS

### TARGET CONTOURED TENNIS ELBOW WRAP

- > Contoured design places pressure over affected tendons while preserving complete range of motion of the elbow
- > Indicated for treatment of medial or lateral epicondylitis
- > Manufactured from latex-free soft loop neoprene



#### PRODUCT NUMBER

	UNIV	S	M	L	XL
1/8"	88-3010	88-3011	88-3012	88-3013	88-3014

#### SIZING GUIDE

	UNIV	S	M	L	XL
	8"-12"	8"-9"	9"-10"	10"-11"	11"-12"

Measurement taken at widest part of forearm

### APPLICATION INSTRUCTIONS

1. Open strap and place padded portion directly over affected area. If applicable, place curvature portion so that it faces elbow joint, and secure.
2. Thread strap through buckle and pull back to fasten hook and loop closure.
3. Adjust for comfort without inhibiting circulation.

### CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Remove any metal stays if possible, close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

### HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



**TELEPHONE**

1-800-426-7353, Ext1



**FAX**

1-603-623-4111



**E-MAIL**

Service@corflex.com



**MAIL**

CORFLEX, Inc.  
Attn: Customer Service  
669 East Industrial Park Dr  
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109  
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.