



CORFLEX

**CORFLEX IS A GLOBALLY BASED
MANUFACTURER AND DISTRIBUTOR OF QUALITY
ORTHOPEDIC AND REHABILITATION PRODUCTS**



TENNIS ELBOW SUPPORTS

TARGET UNIVERSAL TENNIS ELBOW WRAP

- > Features easy wrap around design and available with or without pad; universal design allows for inventory reduction
- > Indicated for treatment of medial or lateral epicondylitis
- > Manufactured from premium grade latex-free neoprene



PRODUCT NUMBER

UNIV	
1/8"	88-3020
1/8" W/Pad	88-3021

SIZING GUIDE

UNIV
8"-12"

Measurement taken at widest part of forearm

APPLICATION INSTRUCTIONS

1. Open strap and place over affected area just below elbow joint.
2. Thread strap through buckle and pull back to fasten hook and loop closure onto itself
3. Adjust for comfort without inhibiting circulation

CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



TELEPHONE

1-800-426-7353, Ext1



FAX

1-603-623-4111



E-MAIL

Service@corflex.com



MAIL

CORFLEX, Inc.
Attn: Customer Service
669 East Industrial Park Dr
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.