



CORFLEX

**CORFLEX IS A GLOBALLY BASED
MANUFACTURER AND DISTRIBUTOR OF QUALITY
ORTHOPEDIC AND REHABILITATION PRODUCTS**



TENNIS ELBOW SUPPORTS

TARGET TENNIS ELBOW SLEEVE W/PAD

- > Features neoprene pad with circumferential strap for added compression over affected tendons
- > Indicated for treatment of medial or lateral epicondylitis
- > Manufactured from premium grade latex-free neoprene



PRODUCT NUMBER

	S	M	L	XL	2XL
1/8"	88-3033	88-3034	88-3035	88-3036	88-3037
3/16"	88-3043	88-3044	88-3045	88-3046	88-3047

SIZING GUIDE

S	M	L	XL	2XL
8"-9"	9"-10"	10"-11"	11"-12"	12"-14"

Measurement taken at widest part of forearm

APPLICATION INSTRUCTIONS

1. With strap open, grasp contoured portion of sleeve and slide over forearm.
2. Make sure pad is placed over affected area of forearm.
3. Pull strap through buckle and pull back onto itself to fasten hook and loop closure.
4. Adjust to desired tension or comfort level.

CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Remove any metal stays if possible, close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



TELEPHONE

1-800-426-7353, Ext1



FAX

1-603-623-4111



E-MAIL

Service@corflex.com



MAIL

CORFLEX, Inc.
Attn: Customer Service
669 East Industrial Park Dr
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.