



**CORFLEX**

**CORFLEX IS A GLOBALLY BASED  
MANUFACTURER AND DISTRIBUTOR OF QUALITY  
ORTHOPEDIC AND REHABILITATION PRODUCTS**



## ELBOW SLEEVES

### TARGET ELBOW SLEEVE W/PAD

- > Provides support, warmth and compression for the elbow with additional pad over elbow for added protection
- > Indicated for treatment of elbow injuries and athletic protection
- > Manufactured from premium grade latex-free neoprene



### PRODUCT NUMBER

9"	S	M	L	XL	2XL
1/8"	88-3063	88-3064	88-3065	88-3066	88-3067
3/16"	88-3163	88-3164	88-3165	88-3166	88-3167

### SIZING GUIDE

S	M	L	XL	2XL
8"-9"	9"-10"	10"-11"	11"-12"	12"-14"

Measurement taken at widest part of forearm

### APPLICATION INSTRUCTIONS

1. With black outside exposed insert arm into the largest end of support and slide over elbow.
2. Make sure seam is on inside of arm with pad over bend of elbow.
3. Adjust placement as required.

### CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

### HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



**TELEPHONE**

1-800-426-7353, Ext1



**FAX**

1-603-623-4111



**E-MAIL**

Service@corflex.com



**MAIL**

CORFLEX, Inc.  
Attn: Customer Service  
669 East Industrial Park Dr  
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109  
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.