



CORFLEX

**CORFLEX IS A GLOBALLY BASED
MANUFACTURER AND DISTRIBUTOR OF QUALITY
ORTHOPEDIC AND REHABILITATION PRODUCTS**



PATELLA STABILIZER SUPPORTS / NEOPRENE

PATELLA STABILIZER W/COR-TRAK BUTTRESS

- > Features movable U-shaped Cor-Trak buttress with textured finish for maximum patella stabilization and spiral stays for moderate medial/lateral support
- > Indicated for patella subluxation, patello-femoral tracking syndrome, and as part of a rehab program for patella tendon injuries
- > Manufactured from premium grade latex-free neoprene with compression molded textured neoprene buttress



PRODUCT NUMBER

13"	XS	S	M	L	XL
1/8"	88-5012	88-5013	88-5014	88-5015	88-5016
1/8" Op Pop	88-5022	88-5023	88-5024	88-5025	88-5026

13"	2XL	3XL	4XL
1/8"	88-5017	88-5018	88-5019
1/8" Op Pop	88-5027	88-5028	88-5029

13"	XS	S	M	L	XL
3/16"	88-5052	88-5053	88-5054	88-5055	88-5056
3/16" Op Pop	88-5062	88-5063	88-5064	88-5065	88-5066

13"	2XL	3XL	4XL
3/16"	88-5057	88-5058	88-5059
3/16" Op Pop	88-5067	88-5068	88-5069

SIZING GUIDE

	6" ABOVE MID-PATELLA	MID-PATELLA	6" BELOW MID-PATELLA
X-Small	16"-18"	12"-13"	11"-12"
Small	17"-19"	13"-15"	12"-13"
Medium	18"-20"	14"-15"	13"-14"
Large	19"-21"	15"-16"	14"-15"
X-Large	20"-22"	16"-18"	15"-16"
2X-Large	21"-23"	18"-20"	16"-17"
3X-Large	24"-26"	20"-22"	17"-18"
4X-Large	27"-29"	22"-24"	18"-19"

APPLICATION INSTRUCTIONS

Note: U-shaped buttress is removable and should be placed where support is needed.

1. Unfasten and loosen the straps at the thigh and calf.
2. With black outside exposed, grasp the edge of the wider portion of the sleeve, keeping the seam positioned to posterior of leg.
3. Insert foot into sleeve and pull onto leg.

CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Remove any metal stays if possible, close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

669 East Industrial Park Drive - Manchester, NH 03109
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.



CORFLEX

**CORFLEX IS A GLOBALLY BASED
MANUFACTURER AND DISTRIBUTOR OF QUALITY
ORTHOPEDIC AND REHABILITATION PRODUCTS**



PATELLA STABILIZER SUPPORTS / NEOPRENE

- 4. Position the sleeve so that stays or hinges are parallel to each other on the side of leg, and the top of the front cut out aligns with the top of the kneecap (patella) and rear cut out is centered in the back of the knee (popliteal) as required.
- 5. Position the support so the buttress is in contact with the kneecap (patella) to provide support.
- 6. Sleeve should sit on leg with approximately 6" of material above mid kneecap (patella).
- 7. Fasten the support straps at the thigh and calf as required for comfortable fit without inhibiting circulation.

HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



TELEPHONE

1-800-426-7353, Ext1



FAX

1-603-623-4111



E-MAIL

Service@corflex.com



MAIL

CORFLEX, Inc.
Attn: Customer Service
669 East Industrial Park Dr
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.