



**CORFLEX**

**CORFLEX IS A GLOBALLY BASED  
MANUFACTURER AND DISTRIBUTOR OF QUALITY  
ORTHOPEDIC AND REHABILITATION PRODUCTS**



**HINGED KNEE SUPPORTS / NEOPRENE**

**PATELLA STABILIZER W/COR-TRAK BUTTRESS & HINGE**

- > Features movable U-shaped Cor-Trak buttress with textured finish for maximum patella stabilization and heavy-duty hinges for optimal medial/lateral support
- > Indicated for medial/lateral instability, muscle/ligament weakness or injury, and degenerative joint disease where patella stabilization is also required
- > Manufactured from premium grade latex-free neoprene with compression molded textured neoprene buttress



**PRODUCT NUMBER**

3/16"	XS	S	M	L	XL	2XL
13"	88-5072	88-5073	88-5074	88-5075	88-5076	88-5077
13" Op Pop	88-5082	88-5083	88-5084	88-5085	88-5086	88-5087

1/8"	XS	S	M	L	XL	2XL
13"	88-5032	88-5033	88-5034	88-5035	88-5036	88-5037
13" Op Pop	88-5042	88-5043	88-5044	88-5045	88-5046	88-5047

**SIZING GUIDE**

	6" ABOVE MID-PATELLA	MID-PATELLA	6" BELOW MID-PATELLA
X-Small	16"-18"	12"-13"	11"-12"
Small	17"-19"	13"-14"	12"-13"
Medium	18"-20"	14"-15"	13"-14"
Large	19"-21"	15"-16"	14"-15"
X-Large	20"-22"	16"-18"	15"-16"
2X-Large	21"-23"	18"-20"	16"-17"
3X-Large	24"-26"	20"-22"	17"-18"
4X-Large	27"-29"	22"-24"	18"-19"

Also available in 3X & 4X

**APPLICATION INSTRUCTIONS**

Note: U-shaped buttress is removable and should be placed where support is needed.

1. Unfasten and loosen the straps at the thigh and calf.
2. With black outside exposed, grasp the edge of the wider portion of the sleeve, keeping the seam positioned to posterior of leg.
3. Insert foot into sleeve and pull onto leg.
4. Position the sleeve so that stays or hinges are parallel to each other on the side of leg, and the top of the front cut out aligns with the top of the kneecap (patella) and rear cut out is centered in the back of the knee (popliteal) as required.
5. Position the support so the buttress is in contact with the kneecap (patella) to provide support.
6. Sleeve should sit on leg with approximately 6" of material above mid kneecap (patella).
7. Fasten the support straps at the thigh and calf as required for comfortable fit without inhibiting circulation.

**CARE & CLEANING**

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Remove any metal stays if possible, close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

**HOW TO PLACE AN ORDER**

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



**TELEPHONE**

1-800-426-7353, Ext1



**FAX**

1-603-623-4111



**E-MAIL**

Service@corflex.com



**MAIL**

CORFLEX, Inc.  
Attn: Customer Service  
669 East Industrial Park Dr  
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109  
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.