



CORFLEX

**CORFLEX IS A GLOBALLY BASED
MANUFACTURER AND DISTRIBUTOR OF QUALITY
ORTHOPEDIC AND REHABILITATION PRODUCTS**



PATELLA STABILIZER SUPPORTS / NEOPRENE

U-SHAPED PATELLA STABILIZER

- > Features sewn-in U-shaped solid tubular buttress to provide optimal contact and patella stability with spiral stays to prevent rollover; available with either superior or inferior buttress placement
- > Indicated for patella subluxation, patello-femoral tracking syndrome, treatment of patella alta/baja and as part of a rehab program for patella tendon injuries
- > Manufactured from premium grade latex-free neoprene



PRODUCT NUMBER

13"	XS	S	M	L
1/8" Inferior U	88-5092	88-5093	88-5094	88-5095
1/8" Inferior U Op Pop	88-5102	88-5103	88-5104	88-5105
13"	XL	2XL	3XL	4XL
1/8" Inferior U	88-5096	88-5097	88-5098	88-5099
1/8" Inferior U Op Pop	88-5106	88-5107	88-5108	88-5109

13"	XS	S	M	L
3/16" Inferior U	88-5132	88-5133	88-5134	88-5135
3/16" Inferior U Op Pop	88-5142	88-5143	88-5144	88-5145
13"	XL	2XL	3XL	4XL
3/16" Inferior U	88-5136	88-5137	88-5138	88-5139
3/16" Inferior U Op Pop	88-5146	88-5147	88-5148	88-5149

13"	XS	S	M	L
3/16" Superior U	88-6202	88-6203	88-6204	88-6205
3/16" Superior U Op Pop	88-6212	88-6213	88-6214	88-6215
13"	XL	2XL	3XL	4XL
3/16" Superior U	88-6206	88-6207	88-6208	88-6209
3/16" Superior U Op Pop	88-6216	88-6217	88-6218	88-6219

SIZING GUIDE

	6" ABOVE MID-PATELLA	MID-PATELLA	6" BELOW MID-PATELLA
X-Small	16"-18"	12"-13"	11"-12"
Small	17"-19"	13"-15"	12"-13"
Medium	18"-20"	14"-15"	13"-14"
Large	19"-21"	15"-16"	14"-15"
X-Large	20"-22"	16"-18"	15"-16"
2X-Large	21"-23"	18"-20"	16"-17"
3X-Large	24"-26"	20"-22"	17"-18"
4X-Large	27"-29"	22"-24"	18"-19"

APPLICATION INSTRUCTIONS

Note: U-shaped buttress is removable and should be placed where support is needed.

1. Unfasten and loosen the straps at the thigh and calf.
2. With black outside exposed, grasp the edge of the wider portion of the sleeve, keeping the seam positioned to posterior of leg.
3. Insert foot into sleeve and pull onto leg.

CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Remove any metal stays if possible, close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

669 East Industrial Park Drive - Manchester, NH 03109
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.



CORFLEX

**CORFLEX IS A GLOBALLY BASED
MANUFACTURER AND DISTRIBUTOR OF QUALITY
ORTHOPEDIC AND REHABILITATION PRODUCTS**



PATELLA STABILIZER SUPPORTS / NEOPRENE

- 4. Position the sleeve so that stays or hinges are parallel to each other on the side of leg, and the top of the front cut out aligns with the top of the kneecap (patella) and rear cut out is centered in the back of the knee (popliteal) as required.
- 5. Position the support so the buttress is in contact with the kneecap (patella) to provide support.
- 6. Sleeve should sit on leg with approximately 6" of material above mid kneecap (patella).
- 7. Fasten the support straps at the thigh and calf as required for comfortable fit without inhibiting circulation.

HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



TELEPHONE

1-800-426-7353, Ext1



FAX

1-603-623-4111



E-MAIL

Service@corflex.com



MAIL

CORFLEX, Inc.
Attn: Customer Service
669 East Industrial Park Dr
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.