



CORFLEX

**CORFLEX IS A GLOBALLY BASED
MANUFACTURER AND DISTRIBUTOR OF QUALITY
ORTHOPEDIC AND REHABILITATION PRODUCTS**



HINGED KNEE SUPPORTS / NEOPRENE

U-SHAPED PATELLA STABILIZER W/HINGE

- > Features sewn-in U-shaped solid tubular buttress for patella stabilization with heavy-duty hinges for optimal medial/lateral support
- > Indicated for medial/lateral instability, muscle/ligament weakness or injury, treatment of patella baja and degenerative joint disease where patella stabilization is also required
- > Manufactured from premium grade latex-free neoprene



Also available in 3X & 4X

PRODUCT NUMBER

3/16"	XS	S	M	L	XL	2XL
13"	88-5152	88-5153	88-5154	88-5155	88-5156	88-5157
13" Op Pop	88-5162	88-5163	88-5164	88-5165	88-5166	88-5167

1/8"	XS	S	M	L	XL	2XL
13"	88-5112	88-5113	88-5114	88-5115	88-5116	88-5117
13" Op Pop	88-5122	88-5123	88-5124	88-5125	88-5126	88-5127

SIZING GUIDE

	6" ABOVE MID-PATELLA	MID-PATELLA	6" BELOW MID-PATELLA
X-Small	16"-18"	12"-13"	11"-12"
Small	17"-19"	13"-14"	12"-13"
Medium	18"-20"	14"-15"	13"-14"
Large	19"-21"	15"-16"	14"-15"
X-Large	20"-22"	16"-18"	15"-16"
2X-Large	21"-23"	18"-20"	16"-17"
3X-Large	24"-26"	20"-22"	17"-18"
4X-Large	27"-29"	22"-24"	18"-19"

APPLICATION INSTRUCTIONS

1. Unfasten and loosen the straps.
2. Grasp the edge of the wider portion of the sleeve, keeping the seam positioned to posterior of leg.
3. Insert foot into sleeve and pull onto leg.
4. Position the sleeve so that stays or hinges are parallel to each other on the side of leg, and the top of the front cut out aligns with the top of the kneecap (patella) and rear cut out is centered in the back of the knee (popliteal) as required.
5. Position the support so the buttress is in contact with the kneecap (patella) to provide support.
6. Sleeve should sit on leg with approximately 6" of material above mid kneecap (patella).
7. Fasten the support straps at the thigh and calf as required for comfortable fit without inhibiting circulation.

CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Remove any metal stays if possible, close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



TELEPHONE

1-800-426-7353, Ext1



FAX

1-603-623-4111



E-MAIL

Service@corflex.com



MAIL

CORFLEX, Inc.
Attn: Customer Service
669 East Industrial Park Dr
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.