



CORFLEX.

**CORFLEX IS A GLOBALLY BASED
MANUFACTURER AND DISTRIBUTOR OF QUALITY
ORTHOPEDIC AND REHABILITATION PRODUCTS**



PATELLA STABILIZER SUPPORTS / NEOPRENE

J-PULL PATELLA STABILIZER

- > Features sewn-in J-shaped solid tubular buttress with adjustable pressure strap that controls pressure applied to patella and spiral stays to prevent roll over
- > Indicated for patella instability, lateral patella subluxation and dislocation
- > Manufactured from premium grade latex-free neoprene



PRODUCT NUMBER

3/16"	XS	S	M	L	XL
13" Right	88-6052	88-6053	88-6054	88-6055	88-6056
13" Left	88-6062	88-6063	88-6064	88-6065	88-6066

3/16"	2XL	3XL	4XL
13" Right	88-6057	88-6058	88-6059
13" Left	88-6067	88-6068	88-6069

3/16"	XS	S	M	L	XL
13" Rt Op Pop	88-6072	88-6073	88-6074	88-6075	88-6076
13" Lf Op Pop	88-6082	88-6083	88-6084	88-6085	88-6086

3/16"	2XL	3XL	4XL
13" Rt Op Pop	88-6077	88-6078	88-6079
13" Lf Op Pop	88-6087	88-6088	88-6089

SIZING GUIDE

	6" ABOVE MID-PATELLA	MID-PATELLA	6" BELOW MID-PATELLA
X-Small	16"-18"	12"-13"	11"-12"
Small	17"-19"	13"-14"	12"-13"
Medium	18"-20"	14"-15"	13"-14"
Large	19"-21"	15"-16"	14"-15"
X-Large	20"-22"	16"-18"	15"-16"
2X-Large	21"-23"	18"-20"	16"-17"
3X-Large	24"-26"	20"-22"	17"-18"
4X-Large	27"-29"	22"-24"	18"-19"

APPLICATION INSTRUCTIONS

1. Release tabs on neoprene overlay over J-Pull.
2. Insert foot into the largest end of support and slide over knee, making sure J-buttress is on the outside of knee.
3. Seam should be placed posteriorly, in back of leg and your kneecap (patella) should be centered in the circular opening.
4. Pull both tabs over J-buttress and attach to loop portion at desired comfort level.

CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



TELEPHONE

1-800-426-7353, Ext1



FAX

1-603-623-4111



E-MAIL

Service@corflex.com



MAIL

CORFLEX, Inc.
Attn: Customer Service
669 East Industrial Park Dr
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.