



**CORFLEX.**

**CORFLEX IS A GLOBALLY BASED  
MANUFACTURER AND DISTRIBUTOR OF QUALITY  
ORTHOPEDIC AND REHABILITATION PRODUCTS**



**PATELLA STABILIZER SUPPORTS / NEOPRENE**

**C-PULL PATELLA STABILIZER**

- > Features sewn-in C-shaped solid tubular buttress with adjustable pressure strap that controls pressure applied to patella and spiral stays to prevent roll over
- > Indicated for patella instability, lateral patella subluxation and dislocation
- > Manufactured from premium grade latex-free neoprene



**PRODUCT NUMBER**

3/16"	XS	S	M	L	XL
13" Right	88-7052	88-7053	88-7054	88-7055	88-7056
13" Left	88-7062	88-7063	88-7064	88-7065	88-7066

3/16"	2XL	3XL	4XL
13" Right	88-7057	88-7058	88-7059
13" Left	88-7067	88-7068	88-7069

3/16"	XS	S	M	L	XL
13" Rt Op Pop	88-7072	88-7073	88-7074	88-7075	88-7076
13" Lf Op Pop	88-7082	88-7083	88-7084	88-7085	88-7086

3/16"	2XL	3XL	4XL
13" Rt Op Pop	88-7077	88-7078	88-7079
13" Lf Op Pop	88-7087	88-7088	88-7089

**SIZING GUIDE**

	6" ABOVE MID-PATELLA	MID-PATELLA	6" BELOW MID-PATELLA
X-Small	16"-18"	12"-13"	11"-12"
Small	17"-19"	13"-14"	12"-13"
Medium	18"-20"	14"-15"	13"-14"
Large	19"-21"	15"-16"	14"-15"
X-Large	20"-22"	16"-18"	15"-16"
2X-Large	21"-23"	18"-20"	16"-17"
3X-Large	24"-26"	20"-22"	17"-18"
4X-Large	27"-29"	22"-24"	18"-19"

**APPLICATION INSTRUCTIONS**

1. Release tabs on neoprene overlay over C-Pull.
2. Insert foot into the largest end of support and slide over knee, making sure C-Buttress is on the outside of knee.
3. Seam should be placed posteriorly, in back of leg and your kneecap (patella) should be centered in the circular opening.
4. Pull both tabs over C-Buttress and attach to loop portion at

**CARE & CLEANING**

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

669 East Industrial Park Drive - Manchester, NH 03109  
Phone (603) 623-3344 - Fax (603) 623-4111 - [www.corflex.com](http://www.corflex.com)

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.



**CORFLEX.**  
.....

**CORFLEX IS A GLOBALLY BASED  
MANUFACTURER AND DISTRIBUTOR OF QUALITY  
ORTHOPEDIC AND REHABILITATION PRODUCTS**



**PATELLA STABILIZER SUPPORTS / NEOPRENE**

desired comfort level.

**HOW TO PLACE AN ORDER**

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



**TELEPHONE**

1-800-426-7353, Ext1



**FAX**

1-603-623-4111



**E-MAIL**

Service@corflex.com



**MAIL**

CORFLEX, Inc.  
Attn: Customer Service  
669 East Industrial Park Dr  
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109  
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.