



CORFLEX.

**CORFLEX IS A GLOBALLY BASED
MANUFACTURER AND DISTRIBUTOR OF QUALITY
ORTHOPEDIC AND REHABILITATION PRODUCTS**



PATELLA STABILIZER SUPPORTS / NEOPRENE

KNEE-O-TRAKKER W/TILT STRAP

- > Features textured C-shaped buttress for maximum patella control with tilt strap to assist in proper tracking and versatile strapping system for dynamic treatment with adjustable tension control
- > Indicated for patella subluxation, patello-femoral syndrome, and as part of a rehab program for patella tendon injuries
- > Manufactured from premium grade latex-free neoprene with compression molded textured neoprene buttress



PRODUCT NUMBER

8"	XS	S	M	L	XL
3/16"	88-7252	88-7253	88-7254	88-7255	88-7256

8"	2XL	3XL	4XL
3/16"	88-7257	88-7258	88-7259

SIZING GUIDE

XS	S	M	L	XL
12"-13"	13"-14"	14"-15"	15"-16"	16"-18"

2XL	3XL	4XL
18"-20"	20"-22"	22"-24"

Measurement taken at mid-patella

APPLICATION INSTRUCTIONS

1. Loosen straps, making sure they remain threaded through d-rings.
2. After determining side of kneecap (patella) that requires support, position buttress on affected side and slide sleeve over knee.
3. Make sure seam is placed posteriorly in back of leg.
4. Your kneecap (patella) should be centered in the circular opening with buttress placed on rim of the kneecap (patella).
5. Grasp both straps by hook tip and pull back onto loop portion of strap.
6. Adjust straps to desired tension or comfort level.

CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



TELEPHONE

1-800-426-7353, Ext1



FAX

1-603-623-4111



E-MAIL

Service@corflex.com



MAIL

CORFLEX, Inc.
Attn: Customer Service
669 East Industrial Park Dr
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.