



CORFLEX

**CORFLEX IS A GLOBALLY BASED
MANUFACTURER AND DISTRIBUTOR OF QUALITY
ORTHOPEDIC AND REHABILITATION PRODUCTS**



HINGED KNEE SUPPORTS / NEOPRENE

KNEE-O-TRAKKER W/HINGE

- > Features textured C-shaped buttress for maximum patella control, versatile strapping system for dynamic treatment with tension control with heavy-duty hinges for added support
- > Indicated for patella subluxation, patello-femoral syndrome and as part of a rehab program for patella tendon injuries where medial/lateral support is required
- > Manufactured from premium grade latex-free neoprene with compression molded textured neoprene buttress



PRODUCT NUMBER

3/16"	XS	S	M	L	XL
13"	88-7272	88-7273	88-7274	88-7275	88-7276

3/16"	2XL	3XL	4XL
13"	88-7277	88-7278	88-7279

SIZING GUIDE

	6" ABOVE MID-PATELLA	MID-PATELLA	6" BELOW MID-PATELLA
X-Small	16"-18"	12"-13"	11"-12"
Small	17"-19"	13"-14"	12"-13"
Medium	18"-20"	14"-15"	13"-14"
Large	19"-21"	15"-16"	14"-15"
X-Large	20"-22"	16"-18"	15"-16"
2X-Large	21"-23"	18"-20"	16"-17"
3X-Large	24"-26"	20"-22"	17"-18"
4X-Large	27"-29"	22"-24"	18"-19"

APPLICATION INSTRUCTIONS

1. Grasp top of support (largest end) and loosen straps.
2. After determining side of kneecap (patella) that requires support, position buttress on affected side and slide sleeve over knee. (Note: buttress may be removed and placed on opposite side.)
3. Make sure seam is placed posteriorly in back of leg and hinges or spiral stays are placed medial/lateral on leg.
4. Your kneecap (patella) should be centered in the circular opening with buttress placed on rim of the kneecap (patella).
5. Grasp both small straps above and below kneecap by hook tip and pull back onto loop portion of strap.
6. Adjust small straps to desired tension or comfort level.
7. Starting with large strap nearest foot, secure strap thru buckle and tighten to desired comfort level. Secure remaining strap in same manner.

CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Remove any metal stays if possible, close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



TELEPHONE

1-800-426-7353, Ext1



FAX

1-603-623-4111



E-MAIL

Service@corflex.com



MAIL

CORFLEX, Inc.
Attn: Customer Service
669 East Industrial Park Dr
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.