



CORFLEX

**CORFLEX IS A GLOBALLY BASED
MANUFACTURER AND DISTRIBUTOR OF QUALITY
ORTHOPEDIC AND REHABILITATION PRODUCTS**



HINGED KNEE SUPPORTS / NEOPRENE

POSTERIOR ADJUSTABLE KNEE SLEEVE W/COR-TRAK BUTTRESS & HINGE

- > Unique design features a sleeve below knee with adjustable wrap above knee, moveable U-shaped Cor-Trak buttress for patella stabilization and heavy-duty hinges for medial/lateral support
- > Indicated for medial/lateral instability, muscle/ligament weakness or injury, ACL/PCL moderate instability, strain/sprains, degenerative joint disease where patella stabilization is required
- > Manufactured from premium latex-free neoprene with compression molded textured neoprene buttress



PRODUCT NUMBER

13"	S	M	L	XL	2XL
3/16"	88-8053	88-8054	88-8055	88-8056	88-8057
1/8" Perf	87-8153	87-8154	87-8155	87-8156	87-8157

13"	3XL	4XL
3/16"	88-8058	88-8059
1/8" Perf	87-8158	87-8159

SIZING GUIDE

	6" ABOVE MID-PATELLA	MID-PATELLA	6" BELOW MID-PATELLA
Small	16"-18"	12"-14"	11"-12"
Medium	18"-19"	14"-16"	13"-14"
Large	19"-22"	16"-18"	15"-16"
X-Large	22"-24"	18"-20"	17"-18"
2X-Large	24"-26"	20"-22"	19"-20"
3X-Large	26"-28"	22"-24"	21"-22"
4X-Large	28"-30"	24"-26"	23"-24"

APPLICATION INSTRUCTIONS

1. Open posterior closure portion of the brace.
2. Slide foot through the sleeve and place so patella opening is over knee cap. Make sure center axis of hinge is aligned with the center of the knee joint.
3. Secure top portion behind the knee to desired comfort level.
4. Starting with the strap nearest foot, secure strap through buckle, tighten to desired comfort level. Attach remaining straps in the same manner.

CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Remove any metal stays if possible, close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



TELEPHONE

1-800-426-7353, Ext1



FAX

1-603-623-4111



E-MAIL

Service@corflex.com



MAIL

CORFLEX, Inc.
Attn: Customer Service
669 East Industrial Park Dr
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.