



CORFLEX

**CORFLEX IS A GLOBALLY BASED
MANUFACTURER AND DISTRIBUTOR OF QUALITY
ORTHOPEDIC AND REHABILITATION PRODUCTS**



HINGED KNEE SUPPORTS / NEOPRENE

POSTERIOR ADJUSTABLE KNEE SLEEVE W/HINGE

- > Unique design offers the convenience of a sleeve below the knee and benefits of an adjustable wrap above the knee with heavy-duty hinges for optimal medial/lateral support; available in two lengths
- > Indicated for medial/lateral instability, muscle/ligament weakness or injury, ACL/PCL moderate instability, strains or sprains and degenerative joint disease
- > Manufactured from premium grade latex-free neoprene



PRODUCT NUMBER

3/16"	S	M	L	XL	2XL
13"	88-8083	88-8084	88-8085	88-8086	88-8087
16"	88-8203	88-8204	88-8205	88-8206	88-8207

3/16"	3XL	4XL
13"	88-8088	88-8089
16"	88-8208	88-8209

1/8"	S	M	L	XL	2XL
Perf	87-8183	87-8184	87-8185	87-8186	87-8187

1/8"	3XL	4XL
Perf	87-8188	87-8189

SIZING GUIDE

	6" ABOVE MID-PATELLA	MID-PATELLA	6" BELOW MID-PATELLA
Small	16"-18"	12"-14"	11"-12"
Medium	18"-19"	14"-16"	13"-14"
Large	19"-22"	16"-18"	15"-16"
X-Large	22"-24"	18"-20"	17"-18"
2X-Large	24"-26"	20"-22"	19"-20"
3X-Large	26"-28"	22"-24"	21"-22"
4X-Large	28"-30"	24"-26"	23"-24"

APPLICATION INSTRUCTIONS

1. Open posterior closure portion of the brace.
2. Slide foot through the sleeve and place so patella opening is over knee cap. Make sure center axis of hinge is aligned with the center of the knee joint.

CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Remove any metal stays if possible, close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



TELEPHONE

1-800-426-7353, Ext1



FAX

1-603-623-4111



E-MAIL

Service@corflex.com



MAIL

CORFLEX, Inc.
Attn: Customer Service
669 East Industrial Park Dr
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.