



**CORFLEX**

**CORFLEX IS A GLOBALLY BASED  
MANUFACTURER AND DISTRIBUTOR OF QUALITY  
ORTHOPEDIC AND REHABILITATION PRODUCTS**



**PATELLA STABILIZER SUPPORTS / NEOPRENE**

**POSTERIOR ADJUSTABLE KNEE SLEEVE W/COR-TRAK BUTTRESS**

- > Unique design offers the convenience of a sleeve below the knee and benefits of an adjustable wrap above the knee with moveable U-shaped Cor-Trak buttress and spiral stays
- > Indicated for patella subluxation, patello-femoral tracking syndrome, and as part of a rehab program for patella tendon injuries
- > Manufactured from premium grade latex-free neoprene with compression molded textured neoprene buttress



**PRODUCT NUMBER**

13"	S	M	L	XL	2XL
3/16"	88-8093	88-8094	88-8095	88-8096	88-8097
1/8" Perf	87-8193	87-8194	87-8195	87-8196	87-8197

13"	3XL	4XL
3/16"	88-8098	88-8099
1/8" Perf	87-8198	87-8199

**SIZING GUIDE**

	6" ABOVE MID-PATELLA	MID-PATELLA	6" BELOW MID-PATELLA
Small	16"-18"	12"-14"	11"-12"
Medium	18"-19"	14"-16"	13"-14"
Large	19"-22"	16"-18"	15"-16"
X-Large	22"-24"	18"-20"	17"-18"
2X-Large	24"-26"	20"-22"	19"-20"
3X-Large	26"-28"	22"-24"	21"-22"
4X-Large	28"-30"	24"-26"	23"-24"

**APPLICATION INSTRUCTIONS**

1. Open posterior closure portion of the brace.
2. Slide foot through the sleeve and place so patella opening is over kneecap.
3. Place Cor-Trak buttress in desired location.
4. Secure Velcro closure behind the knee to desired comfort level.
5. Starting with strap nearest foot, attach Velcro strap through buckle, tighten to desired comfort level. Attach remaining strap in the same manner.

**CARE & CLEANING**

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Remove any metal stays if possible, close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

**HOW TO PLACE AN ORDER**

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



**TELEPHONE**

1-800-426-7353, Ext1



**FAX**

1-603-623-4111



**E-MAIL**

Service@corflex.com



**MAIL**

CORFLEX, Inc.  
Attn: Customer Service  
669 East Industrial Park Dr  
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109  
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.