



CORFLEX

**CORFLEX IS A GLOBALLY BASED
MANUFACTURER AND DISTRIBUTOR OF QUALITY
ORTHOPEDIC AND REHABILITATION PRODUCTS**



HINGED KNEE SUPPORTS / NEOPRENE

POSTERIOR ADJUSTABLE KNEE SLEEVE W/R.O.M. HINGE

- > Features R.O.M. hinges that allow for full flexion and extension control and unique design that offers the convenience of a sleeve below the knee and benefits of an adjustable wrap above the knee; available in two lengths
- > Indicated for medial/lateral instability, muscle/ligament weakness or injury, ACL/PCL moderate instability, strains or sprains, and degenerative joint disease where extension/flexion control is required
- > Manufactured from premium grade latex-free neoprene



PRODUCT NUMBER

3/16"	S	M	L	XL	2XL
13"	88-8133	88-8134	88-8135	88-8136	88-8137
16"	88-8213	88-8214	88-8215	88-8216	88-8217

3/16"	3XL	4XL
13"	88-8138	88-8139
16"	88-8218	88-8219

SIZING GUIDE

	6" ABOVE MID-PATELLA	MID-PATELLA	6" BELOW MID-PATELLA
Small	16"-18"	12"-14"	11"-12"
Medium	18"-19"	14"-16"	13"-14"
Large	19"-22"	16"-18"	15"-16"
X-Large	22"-24"	18"-20"	17"-18"
2X-Large	24"-26"	20"-22"	19"-20"
3X-Large	26"-28"	22"-24"	21"-22"
4X-Large	28"-30"	24"-26"	23"-24"

APPLICATION INSTRUCTIONS

1. Remove R.O.M. hinge from hinge pocket, open hinge cover, set hinge by placing pins in desired setting, close hinge cover and reinsert into hinge pocket.
2. Open posterior closure portion of the brace.
3. Slide foot through the sleeve and place so patella opening is over kneecap. Make sure center axis of hinge is aligned with the center of the knee joint.
4. Secure top portion behind the knee to desired comfort level.
5. Starting with strap nearest foot, secure strap through buckle, tighten to desired comfort level. Attach remaining strap in the same manner.

CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Remove any metal stays if possible, close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:

TELEPHONE	FAX	E-MAIL	MAIL
1-800-426-7353, Ext1	1-603-623-4111	Service@corflex.com	CORFLEX, Inc. Attn: Customer Service 669 East Industrial Park Dr Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.