



CORFLEX

**CORFLEX IS A GLOBALLY BASED
MANUFACTURER AND DISTRIBUTOR OF QUALITY
ORTHOPEDIC AND REHABILITATION PRODUCTS**



PATELLA STABILIZER SUPPORTS / NEOPRENE

KINETIC POSTERIOR ADJUSTABLE KNEE SLEEVE W/COR-TRAK BUTTRESS

- > Features dynamic buttress for adjustable tension control and spiral stays; unique design offers the convenience of a sleeve below the knee and benefits of an adjustable wrap above the knee
- > Indicated for patella subluxation, patello-femoral tracking syndrome, and correction of patella tilt
- > Manufactured from premium grade latex-free neoprene with compression molded textured neoprene buttress



PRODUCT NUMBER

| 13" | S | M | L | XL | 2XL |
|-------|---------|---------|---------|---------|---------|
| 3/16" | 88-8143 | 88-8144 | 88-8145 | 88-8146 | 88-8147 |

| 13" | 3XL | 4XL |
|-------|---------|---------|
| 3/16" | 88-8148 | 88-8149 |

SIZING GUIDE

| | 6" ABOVE MID-PATELLA | MID-PATELLA | 6" BELOW MID-PATELLA |
|----------|----------------------|-------------|----------------------|
| Small | 16"-18" | 12"-14" | 11"-12" |
| Medium | 18"-19" | 14"-16" | 13"-14" |
| Large | 19"-22" | 16"-18" | 15"-16" |
| X-Large | 22"-24" | 18"-20" | 17"-18" |
| 2X-Large | 24"-26" | 20"-22" | 19"-20" |
| 3X-Large | 26"-28" | 22"-24" | 21"-22" |
| 4X-Large | 28"-30" | 24"-26" | 23"-24" |

APPLICATION INSTRUCTIONS

1. Product is packaged with Kinetic Cor-Trak buttress ready to fit right leg and position patella more medially.
2. If you are fitting left leg, remove tabs and Cor-Trak buttress and place through openings on opposite side.
3. Open posterior closure portion of the brace.
4. Slide foot through the sleeve and place so patella opening is over kneecap.
5. Secure top portion behind the knee to desired comfort level.
6. Starting with strap nearest foot, secure strap through buckle, tighten to desired comfort level. Attach remaining strap in the same manner.
7. Grasp both tabs and activate Kinetic pull to desired pressure, attach tabs.

CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Remove any metal stays if possible, close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



TELEPHONE

1-800-426-7353, Ext1



FAX

1-603-623-4111



E-MAIL

Service@corflex.com



MAIL

CORFLEX, Inc.
Attn: Customer Service
669 East Industrial Park Dr
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.