



CORFLEX.

**CORFLEX IS A GLOBALLY BASED
MANUFACTURER AND DISTRIBUTOR OF QUALITY
ORTHOPEDIC AND REHABILITATION PRODUCTS**



PATELLA STABILIZER SUPPORTS / NEOPRENE

MEDIAL-LATERAL PATELLA STABILIZER

- > Features sewn-in medial-lateral tubular buttress to provide optimal contact and patella stability with spiral stays to prevent roll over
- > Indicated for patella subluxation, patello-femoral tracking syndrome, and as part of a rehab program for patella tendon injuries
- > Manufactured from premium grade latex-free neoprene



PRODUCT NUMBER

13"	XS	S	M	L	XL
3/16"	88-8182	88-8183	88-8184	88-8185	88-8186

13"	2XL	3XL	4XL
3/16"	88-8187	88-8188	88-8189

SIZING GUIDE

	6" ABOVE MID-PATELLA	MID-PATELLA	6" BELOW MID-PATELLA
X-Small	16"-18"	12"-13"	11"-12"
Small	17"-19"	13"-14"	12"-13"
Medium	18"-20"	14"-15"	13"-14"
Large	19"-21"	15"-16"	14"-15"
X-Large	20"-22"	16"-18"	15"-16"
2X-Large	21"-23"	18"-20"	16"-17"
3X-Large	24"-26"	20"-22"	17"-18"
4X-Large	27"-29"	22"-24"	18"-19"

APPLICATION INSTRUCTIONS

1. Unfasten and loosen the straps at the thigh and calf.
2. Grasp the edge of the wider portion of the support and pull onto leg.
3. Position the support so that the medial-lateral buttress is positioned around sides of the kneecap (patella).
4. Fasten the support straps at the thigh and calf so that they fit comfortably around the knee without inhibiting circulation

CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Remove any metal stays if possible, close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



TELEPHONE

1-800-426-7353, Ext1



FAX

1-603-623-4111



E-MAIL

Service@corflex.com



MAIL

CORFLEX, Inc.
Attn: Customer Service
669 East Industrial Park Dr
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.